Bard Parker®

aspensurgical.

## Protected Disposable Scalpel

### Important Usage Guidelines - Non-Sterile

Aspen Surgical Products, Inc.  
6945 Southbelt Drive SE  
Caledonia, MI 49316 USA  
Phone 616-698-7100  
Toll-Free 888-364-7004  
aspensurgical.com

EC REP  
Emergo Europe  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands  
Made in USA

CE  
2797

8365945 | DGP64 | REV3 | 01 DEC 2020

### English

#### Bard-Parker® Protected Disposable Scalpel

#### Important Usage Guidelines

##### Intended Use:

Intended for tissue separation and other procedures that require a sharp blade to puncture or cut.

##### Intended User/ Patient Target Groups:

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

##### Precautions:

- This device is single use only. Reuse of device could result in infection/contamination and/ or device failure which could lead to patient harm.
- Ruler shown on scalpel handle should be used as an indicator only, and not for accurate measurements.
- During use avoid twisting, bending or putting excessive force or strain on the blade in order to help prevent breakage.
- A singular blade is intended to cut tissue up to 24 inches.
- If blade becomes dull or breaks, dispose and replace product.
- If safety shield or housing malfunction, dispose and replace product.

##### Sterilization Instructions:

To ensure SAL of 10<sup>-6</sup>, this product must be sterilized via gamma sterilization method in its original packaging configuration at a sterilization dose range of 9.3–35.0 kGy.

##### To Use

- Depress top button on protective shield.
- Retract shield to rear locked position to fully uncover blade

##### To Cover

- Depress top button on protective shield.
- Advance shield to forward locked position at lock indicator line.

##### Disposal:

Dispose in approved puncture-resistant sharps container according to facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

##### Notice to Users and/or Patients in EU:

Any serious incident that has occurred in relation to the device, should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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