Symmetry Surgical® BOOKWALTER® Horizontal Ring Holder

Symmetry Surgical Inc.
3034 Owen Drive,
Antioch, TN 37013 USA
☎ 1-800-251-3000
Fax: 1-615-964-5566
www.symmetrysurgical.com

Symmetry Surgical GmbH
Maybachstraße 10
78532 Tuttlingen, Germany
☎ +49 7461 96490
Fax: +49 7461 77921

LCN 205182-001/E
© 2006–2015 Symmetry Surgical
♻ Revised 09/15

ENGLISH

IMPORTANT INFORMATION

Please Read Before Use

Symmetry Surgical® BOOKWALTER® Horizontal Ring Holder

Rx Only

General Information

If the instruments are to be used with the BOOKWALTER® Retractor, refer to the instruction manual that provides complete operating instructions and is supplied with the BOOKWALTER® Retractor Kit II (catalog no. 50-4542). Extra copies of this manual are available from Symmetry Surgical, 3034 Owen Drive, Antioch, TN 37013 USA. To obtain extra copies outside the United States, contact your local Symmetry Surgical representative. If these instruments are to be used with any other retractor, please refer to that manufacturer's instruction manual.

Description

The Symmetry Surgical BOOKWALTER® Horizontal Ring Holder (catalog no. 50-4678) is assembled with a post coupling (catalog no. 50-4554) and a table post (catalog no. 50-4581). Refer to package label for a description of contents.

Indications

The BOOKWALTER® Retractor is a self-retaining instrument that provides multi-directional exposure of the abdominal cavity.

WARNINGS

The retractor system can exert a great amount of force on the wound edges and on tissues. Release the retractor for short periods of time or shift the blades to relieve the pressure on any one point.

PRECAUTIONS

If using monopolar electrosurgical equipment while using the BOOKWALTER® Retractor Kit, consult the basic rules of usage concerning electrosurgery (see EN 60 601-2-2), as well as the manufacturer's instructions for use regarding specific warnings and conditions.

Be sure to electrically isolate the patient from (high frequency leakage currents to) the operating table, in accordance with the instructions provided by the manufacturer of the electrosurgical equipment.

Ensure that the patient does not come into contact with metal parts that are earthed or that have an appreciable capacitance to earth (for example: operating table, supports), in accordance with the instructions provided by the manufacturer of the electrosurgical equipment.

Avoid small-area contacts with devices having a low impedance to earth at high frequencies, as unwanted burns may result.

Instructions for Use

Attachment of the Vertical Table Post

Note: Be sure the operating table side rail does **not** have extra material draped over it. The clamp of the vertical table post is designed to hold no more than two double thicknesses of cotton drapes.

Pull out the recessed lever, located at the top of the vertical post (Figures 1 and 2), and fully open the rail clamp by turning the lever in a counter-clockwise direction.

Attach the table post over the drapes as shown in Figure 3. Retighten the clamp to secure the post by turning the lever in a clockwise direction as shown in Figure 4.

Attachment of the Post Coupling and BOOKWALTER® Horizontal Ring Holder

Slide the BOOKWALTER Horizontal Ring Holder into either of the holes in the post coupling, keeping the starburst fully disengaged for accurate positioning. Slide the post coupling and BOOKWALTER Horizontal Ring Holder assembly over the top of the vertical post, to its approximate location, and lightly tighten the coupling on the table post.

Note: To prevent eventual wear of the starburst's teeth, the ring holder must be assembled with the starburst fully disengaged.

Insert the segmented ring into the ring holder's open adjustable jaws and turn the knob clockwise to fully secure the attachment. Make any final adjustments and turn the sliding lever in a clockwise direction to fully engage the starburst's teeth and secure the final position of the assembly (Figure 5).

Note: To avoid slippage of the assembled device, the starburst's teeth must be fully engaged.

Note: The position of the post coupling lever on the vertical post should be convenient to the surgeon. Make any final positioning adjustments before tightening the post coupling.

CAUTION: The post coupling bolt should never be tightened unless the post coupling is attached to the bars. The slight deformity of the circular opening that might result could affect the smooth fit between the post coupling and the bars.

Reprocessing Instructions

Refer to Symmetry Surgical Instructions for Use #204233 for updated information regarding reprocessing the instrument.

Copies of this instruction sheet are available by requesting IFU #204233 from Symmetry Surgical Customer Service at:

Telephone: 800 251 3000

Fax: +1 615 964 5566

E-mail:

customerservice@symmetrysurgical.com

CAUTION: Only the post coupling can be disassembled for cleaning. Do not attempt to disassemble the table post or BOOKWALTER® Horizontal Ring Holder.

IMPORTANT: The vertical table post and post coupling should always be lubricated to keep these parts functional.

CAUTION: To ensure proper function do not lubricate the BOOKWALTER® Horizontal Ring Holder (catalog no. 50-4678).

Service and Repair

For service or repair outside the United States, contact your local Symmetry Surgical representative.

Inside the United States, send instruments for service or repair to:

Symmetry Surgical Inc.
3034 Owen Drive,
Antioch, TN 37013 USA

Always include a repair purchase order number and a written description of the problem. Use only approved authorized dealerships for repairs.

Warranty

Symmetry Surgical warrants that this medical device is free from defects in both materials and workmanship. **Any other express or implied warranties, including warranties of merchantability or fitness, are hereby disclaimed. Suitability for use of this medical device for any particular surgical procedure should be determined by the user in conformance with the manufacturer's instructions for use. There are no warranties that extend beyond the description on the face hereof.**

BOOKWALTER® is a registered trademark of Symmetry Surgical

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