

Instructions for Use: REF# LHP100

1. Remove from packaging and place the positioner at the cephalad end of the O.R. Table.
2. Position the patient so that the top of the head is near the tallest portion of the positioner.
3. Remove any bottom layers necessary to achieve proper alignment of the patient's cervical spine.
4. Ensure that the patient's ear that is in contact with the positioner is oriented in the center of the ear hole.
5. Ensure that the patient's dependent eye is unobstructed, visible and oriented in the center of the eye channel.
6. Discard after use.



The central cutout keeps the ear free of pressure. The patented eye channel allows the provider to constantly monitor the eye pressure to ensure it is free from pressure that could result in catastrophic injury.

The tear-away bottom layers afford the provider the ability to adjust the pillow to patients of varying sizes.

Not made with natural rubber latex



Improvised Positioning



Improper Alignment



Proper Positioning

Reliable Spine Alignment



Manufactured For:

**PRIME
MEDICAL**
1410 E Weisgarber
Knoxville, TN 37909
865-776-8994



EC REP



QNET BV
Kantstraat 19
NL-5076NP Haaren
The Netherlands