

## Instructions for Use: REF# LHP100

1. Remove from packaging and place the positioner at the cephalad end of the O.R. Table.
2. Position the patient so that the top of the head is near the tallest portion of the positioner.
3. Remove any bottom layers necessary to achieve proper alignment of the patient's cervical spine.
4. Ensure that the patient's dependent ear is oriented in the center of the ear hole.
5. Ensure that the patient's dependent eye is oriented in the center of the eye channel.
6. Discard after use.

  
**PRIME**  
MEDICAL LLC



Improvised Positioning



Improper Alignment

Wrap the patient's arms in the "wing" portion of the STP pad, and use the draw sheet to secure the arms by running the sheet over the arms and back underneath the patient. Ensure that the arms are fully protected and secured prior to surgery.



Manufactured For:

  
**PRIME**  
MEDICAL  
5802 Kingston Pike  
Knoxville, TN 37919  
865-776-8994



QNET BV  
Kantstraat 19  
NL-5076NP Haaren  
The Netherlands



If using a standard pillow for head positioning, ensure that the patient's shoulders are in contact with the STP pad and not the pillow.

ALL STP PADS ARE LATEX FREE. DISCARD AFTER USE.

Rev. 12/16