

Precision™ Single-Use Non-Stick Bipolar Forceps

Generator Settings

Reduce the surgeon's normal generator settings by 1/3 to 1/2

- Understand power settings for each surgeon, for each type of case
- Recommend a reduction in settings specific to the type of generator. In general, that will be 1/3 to 1/2 the surgeon's normal settings. Settings can always be turned up if not sufficient.

Generator Type	Power Settings Reduction for Cut/Coagulation
Malis™	1/3
ValleyLab™/Covidien™	1/3
ConMed™	1/2