# Symmetry Surgical® Surgical Instruments with Titanium Nitrate Coating

## Care and Handling

**Symmetry Surgical Inc.**  
3034 Owen Drive,  
Antioch, TN 37013 USA  
☎ 1-800-251-300  
Fax: 1-615-964-5566  
[www.symmetrysurgical.com](http://www.symmetrysurgical.com)

Symmetry Surgical GmbH  
Maybachstraße 10  
78532 Tuttlingen, Germany  
☎ +44-114-285-5881  
Fax: +49 7461 77921

For Class II products, the notified  
body reference number is 0086

**LCN 206345-001/F**  
© 2009–2015 Symmetry Surgical  
♻ Revised 09/15

# ENGLISH

## IMPORTANT INFORMATION

Please Read Before Use

## Symmetry Surgical® Surgical Instruments with Titanium Nitrate Coating

### Care and Handling

These care and handling instructions apply to Symmetry Surgical® Surgical Instruments with titanium nitrate coating. Please telephone Symmetry Surgical Customer Service at 800 251 3000 or fax: 1-615-964-5566 if you need additional assistance.

### WARNINGS

- Follow the instructions and warnings issued by the suppliers of any cleaning and disinfection agents and equipment used.
- Do not exceed 140°C during sterilization and reprocessing.
- Highly alkaline conditions can damage products with aluminum parts.
- Avoid exposure to hypochlorite solutions, as these will promote corrosion and can cause pitting.

### Precautions

- The Symmetry Surgical Surgical Instruments with titanium nitrate coating should be handled with care to eliminate potential for scratching the coating. **DO NOT USE A WIRE BRISTLE BRUSH TO CLEAN.** Use a nylon bristle brush, sponge, or cloth to clean.
- Complex devices, such as those with tubes, hinges, retractable features, mated surfaces, and textured surface finishes, require special attention during cleaning. Manual pre-cleaning of such device features is required before automated cleaning processing.
- It is the responsibility of the reprocessor to ensure that the reprocessing is actually performed using appropriate equipment, materials, and personnel to achieve the desired result. This normally requires validation and routine monitoring of the process. Any deviation by the reprocessor from these instructions should be evaluated for effectiveness and potential adverse consequences.

### Limitations on Reprocessing

- Repeated processing has minimal effects on instrument life and function.
- End of useful life is generally determined by wear or damage in surgical use.
- Carefully inspect instruments between uses to verify proper functioning.

### Decontamination Considerations – Creutzfeldt-Jakob Disease (CJD)

Under certain classifications of risk, the World Health Organization (WHO), or local regulatory authorities recommend special CJD inactivation processing procedures. Consult WHO, your hospital, and local regulations for further information.

## Reprocessing Instructions

### Care at the point of use

- Clean instruments as soon as possible after use. If cleaning must be delayed, immerse instruments in a compatible detergent solution to prevent drying and encrustation of surgical soil.
- Avoid prolonged exposure to saline to minimize the chance of corrosion.
- Remove excessive soil with a disposable wipe.

### A. Manual Cleaning – ALL INSTRUMENTS

- Clean delicate instruments separately from other instruments.
- Disassemble instruments that are intended to come apart for cleaning.
- Prepare an enzymatic cleaning solution in accordance to the manufacturer's instructions.
- Soak soiled instruments for 5 minutes.
- Follow the additional steps in Cleaning sections B., or C., which follow. Select the category that most closely matches the type of instrument.
- Use a nylon bristle brush to remove all traces of blood and debris; pay close attention to any hard to reach areas, textured surfaces, or crevices.
- Rinse the instrument thoroughly with warm tap water.
- Ultrasonically clean the instrument for 10 minutes in neutral pH detergent, prepared in accordance with the manufacturer's instructions.
- Rinse the instrument thoroughly with warm tap water.
- Dry the instrument immediately after final rinse.

### B. Manual Cleaning –

Instruments with cannulations or lumens (i.e. tubes), or holes

- Follow the steps in the section *Manual Cleaning – ALL INSTRUMENTS*.
- When cleaning, use a tight-fitting, soft, non-metallic cleaning brush or pipe cleaner to scrub the cannula, lumen, or hole. Push in and out, using a twisting motion to remove debris. Use a syringe filled with enzymatic cleaning solution to flush hard to reach internal areas.
- When rinsing, pay particular attention to flush the cannulations, lumens, or holes with warm tap water.
- Dry internal areas with filtered compressed air.

### C. Manual Cleaning –

Articulating Instruments (those with moveable parts)

- Follow the steps in the section *Manual Cleaning – ALL INSTRUMENTS*.
- When cleaning, fully immerse the instrument in the cleaning solution to avoid aerosol generation. Brush with a soft non-metallic bristle brush to remove all traces of blood and debris. Pay close attention to threads, crevices, seams, and any hard to reach areas. Actuate any moveable mechanisms, such as hinged joints, box locks, or spring-loaded features, to free trapped blood and debris. If the components of the instrument can be retracted, retract or open the part while cleaning the area. For instruments with flexible shafts, flex the instrument under the cleaning solution.
- When rinsing; pay particular attention to internal areas and moveable parts. Actuate moveable parts while rinsing. If the components of the instrument can be retracted, retract or open the part while rinsing the area. For instruments with flexible shafts, flex the instrument under the rinse solution.
- Dry internal areas with filtered compressed air.

## Automated Cleaning

- For instrument types with complex design features, such as those described in *Manual Cleaning* sections B. and C. above, it is important to manually clean prior to automated processing to improve the removal of adherent soil. Brush the instrument, actuate mechanisms, agitate and/or irrigate under the surface of the cleaning solution to prevent the creation of aerosols.
- Clean, using the “INSTRUMENTS” cycle in a validated washer disinfectant and a pH neutral cleaning agent intended for use in automated cleaning. The cleaning cycle should incorporate enzyme pre-wash, wash, rinse, thermal rinse, and drying steps.
- Load instruments so that hinges are open and cannulations and holes can drain.
- Place heavier instruments on the bottom of containers. Do not place heavy instruments on top of delicate instruments.
- For instruments with concave surfaces, such as curettes, place instrument with the concave surface facing downward to facilitate draining.

## Inspection

- Inspect all instruments before sterilization or storage to ensure the complete removal of soil from surfaces, tubes and holes, moveable parts.
- If areas are difficult to inspect visually, check for blood by immersing or flushing the instrument in a 3% hydrogen peroxide solution. If bubbling is observed, blood is present. **Rinse instruments thoroughly after using hydrogen peroxide solution.**
- If soil is still present, re-clean the instrument.

## Disinfection

Symmetry Surgical instruments must be terminally sterilized prior to surgical use. See *Sterilization* instructions.

## Maintenance

Between uses, lubricate moving parts with a water-soluble lubricant in accordance with the manufacturer's instructions. PRESERVE® Lubricant is recommended for instruments to be steam sterilized (REF 43-1033).

## Inspection and Functional Testing

- Visually inspect the instrument and check for damage and wear.
- Cutting edges should be free of nicks and have a continuous edge.
- Jaws and teeth should align properly.
- Moveable parts should have smooth movement without excessive play.
- Locking mechanisms should fasten securely and close easily.
- Long, thin instruments should be free of bending and distortion.

## Packaging

If desired, use instrument trays to contain instruments that are provided in sets. Double wrap instruments in accordance with local procedures, using standard wrapping techniques such as those described in ANSI/AAMI ST46-1993.

## Sterilization

Use a validated, properly maintained and calibrated steam sterilizer.

Effective steam sterilization can be achieved using the following cycles:

Cycle Type	Temperature	Exposure Time
Prevacuum	132–134°C	4 minutes
Prevacuum	134–137°C	3 minutes

## Storage

Store sterile packaged instruments in a manner that provides protection from dust, moisture, insects, vermin, and extremes of temperature and humidity.

## Warranty

Symmetry Surgical warrants that this medical device is free from defects in both materials and workmanship. **Any other express or implied warranties, including warranties of merchantability or fitness, are hereby disclaimed. Suitability for use of this medical device for any particular surgical procedure should be determined by the user in conformance with the manufacturer's instructions for use. There are no warranties that extend beyond the description on the face hereof. Abuse or misuse of the product or failure to comply with the instructions for use shall void this warranty.**

® PRESERVE is a registered trademark of Symmetry Surgical

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