

Instructions for use

Green Genie Suture Passer 710001

Previously Beatty® reorder # 032401

Intended Use:

Intended to aid in the passing of sutures through bone and soft tissue during orthopedic surgery.

Intended User:

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

Precautions:

- Use caution when pulling suture through the bone and ensure that no soft tissue is caught in the device.
- Do not over fill the wire loop with suture before passing as you risk breaking your bone tunnel while pulling it through.
- Bend wire only.
- This device is single use only.
- Use caution when opening sterile package, device may roll.
- Do not use product if product is damaged or particulate is found in sterile packaging.

Disposal:

Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

Instructions for Use:

1. Used to pass suture through bone (surgical equivalent to a needle threader).
2. Place wire loop through predrilled bone and place suture through the loop and gently pull through the bone.
3. Pull gently through the bone.
4. Discard after use.



Aspen Surgical Products, Inc.

6945 Southbelt Dr. SE, Caledonia, MI 49316

phone +1 616.698.7100 toll-free +1 888.364.7004

aspensurgical.com

Made in USA



STERILE EO

Rx ONLY