Hip Positioners

Non-Sterile

Racine Pelvic Positioner Pads 712800

Carson 8" Perineal Post Pad 712200

Perineal Post Cover 712400

Lateral Positioner Pad 712600

Aspen Surgical Products, Inc.

6945 Southbelt Drive SE

Caledonia, MI 49316 USA

Phone +1 616-698-7100

Toll-Free 888-364-7004

aspensurgical.com

Emergo Europe

Westervoortsedijk 60

6827 AT Arnhem

The Netherlands

For Country of Origin See Product Label

English

Intended Use

Intended to support and maintain the position of an anesthetized patient during surgery.

Intended User/Patient Target Groups

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

Instruction for Use:

- For 712800 and 712600, attach pad set to the base and back plate and support column. Place patient in the lateral position.
- For all other devices, secure patient in place, following table manufacturer's instructions.
- Device is intended for limited contact duration, less than or equal to 8 hours, less if damaged.
- Discard after use.

Compatibility Chart:

IMP® MCGUIRE™ Hip Positioner 1084	Racine Pelvic Positioner Pads 712800
Smith & Nephew® Supine Hip Positioning System 72200624 and Mizuho OSI®, Hana®, ProFx®, and Hana SSXT® Table	Carson 8" Perineal Post Pad 712200
Mizuho OSI®, Hana®, ProFx®, and Hana SSXT® Tables	Perineal Post Cover 712400
IMP® UNIVERSAL LATERAL POSITIONER® 1092	Lateral Positioner Pad 712600

Precautions:

- Ensure pad/cover is secure in the positioning device and that it stays in place while the patient is on the table.
- Take care when positioning patient to avoid any pinching or pressure on the skin.
- Use with approved positioning device.
- Where applicable, ensure pad is centered between patient's legs.
- This device is single use only.
- Do not sterilize.

Warning

Reuse of device could result in infection/contamination and/or device failure which could lead to patient harm.

Disposal

Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional and/or local laws and regulations.

Notice to Users and/or Patients in EU

Any serious incident that has occurred in relation to the device should be reported to Aspen Surgical Inc., the governing body, and the Competent authority of the member state in which the incident occurred.

IMP®, Innovative Medical Products, and McGuire™ Hip Positioner 1084 are registered trademark or trademarks of Medical Products, Inc. ("IMP"). Aspen Surgical Products, Inc. is not affiliated with, endorsed by, or sponsored by IMP. Smith & Nephew® is a registered trademark of Smith & Nephew PLC (Smith & Nephew). Mizuho OSI®, HANA®, PROFX®, SSXT®, are registered trademark or trademarks of Mizuho OSI, Inc. ("Mizuho OSI"). Aspen Surgical Products, Inc. is not affiliated with, endorsed by, or sponsored by Smith & Nephew®, or Mizuho OSI®. IMP® and Universal Lateral Positioner® 1092 are registered trademarks of Innovative Medical Products, Inc. ("IMP"). Aspen Surgical Products, Inc. is not affiliated with, endorsed by, or sponsored by IMP.

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Emergo Consulting (UK) Limited