

## Secure360 Positioning Device


### Instructions for Use

---

1. Position the Secure360 Positioning Device on the Operating Room table by placing the “head” portion of the device flush with the edge of the table and centered between the sides of the surgical table.
2. Feed the Velcro straps through the railing alongside the edge of the surgical table and fasten securely.



3. Place the patient onto the table with their chin cradled in the “head” portion and the endotracheal tube resting along the outside of the positioning device. The patient’s abdomen should be resting along the angled portion of the positioning device and their legs and feet placed on either side of the channel. (The channel can be used to house the catheter tubing.) The patient’s arms should be tucked between the side of the positioning device and alongside the patient’s body.

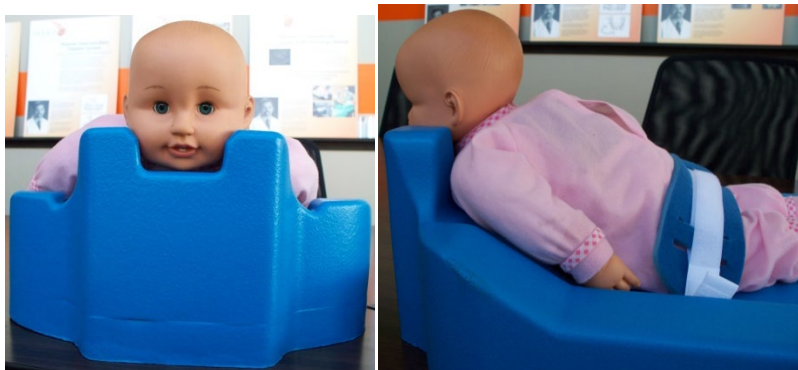

4. Secure the patient onto the table by cradling the patient’s buttocks with the foam strap. Secure the strap in place by fastening the Velcro strap to the Velcro located on the foam strap.

\* The Secure360 Positioning Device is intended for children 6-18 months of age