EN / English	INSTRUCTIONS FOR USE Non-Sterile Single-Use Protective Covers
---------------------	--

Prepare probe/instrument per system’s operational manual. After procedure, to remove the cover follow the steps shown for mounting on probe/instrument, in reverse order.

INTENDED PURPOSE/INTENDED USE

Non-Sterile Probe/Non-Sterile Instrument Cover | These protective covers are medical devices utilized over imaging probes or instruments to provide a sanitary field and minimize cross contamination during diagnostic imaging procedures, operations on the body surface, and intra-cavity and/or intra-operative procedures. Protective covers help prevent transfer of microorganisms, body fluids, and particulate matter to the patient and healthcare provider during procedures using ultrasound transducers/imaging instruments and systems.

Non-Sterile System Drape | This is a medical device used as protective covering for imaging systems, cords, monitors, and keyboards to minimize cross contamination. Protective covers help prevent transfer of microorganisms, body fluids, and particulate matter to the patient and healthcare provider during procedures using ultrasound transducers/imaging instruments and systems.

INTENDED USER

These devices are used by healthcare professionals trained and qualified in using this type of device.

BEFORE PERFORMING PROCEDURE

Verify proper fit in the probe/instrument system. Cover is to be placed tightly over transducer face to eliminate wrinkles and air bubbles, taking care to avoid piercing cover.

WARNING

- This product is single-use only. Do not reuse. Reuse of the single use devices may create a risk of cross contamination, patient infection, and/or malfunction of the device.
- Disposal: After use, dispose of single-use components as infectious waste.
- Do not use if found damaged or does not fit properly. Discard and use an intact package.
- Malfunction: In the case of any signs of a malfunction or changes in performance which may affect safety (e.g., holes or cuts in cover), stop procedure, discard defective cover, use applicable contamination control methods to prevent patient infection, and replace cover with a new one.
- Storage Conditions: Keep away from direct sunlight or extreme temperatures/heat sources. Store in a cool, dry place.
- Other devices intended to be used: Protective covers are used with ultrasound probes or imaging instruments and ultrasound transmission gel (water base).

Notice: Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user is established.

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