The GREENBERG® Retractor

Symmetry Surgical Inc.
3034 Owen Drive
Antioch, TN 37013 USA
☎ 1-800-251-3000
FAX: 1-615-964-5566
www.symmetrysurgical.com

EC REP

Symmetry Surgical GmbH
78532 Tuttlingen, Germany
Maybachstraße 10
☎ +49 7461 96490
Fax: +49 7461 77921

CE 2797

LCN 50-1500GS/C

♻️ 10/2022

ENGLISH**IMPORTANT INFORMATION**

Please Read Before Use

The Greenberg® Retractor System

INSTRUCTIONS FOR USE

Introduction	2
Indications for Use	2
Contraindications	2
PRECAUTIONS	2
Greenberg Simplicity® Components	3
Greenberg Simplicity® General Use	4
Care and Maintenance	7
Ordering Information	7
Warranty	8
Service and Repair	8

Introduction

Indications for Use

The Greenberg® Retractor System is indicated for self-retaining retraction, support of the surgeon's hands, and support of surgical instruments.

Contraindications

The Greenberg® Retractor System is not designed, sold, or intended for use except as indicated.

WARNINGS

Failure to read and follow instructions in this manual may lead to device malfunction, which could result in patient injury or death.

PRECAUTIONS

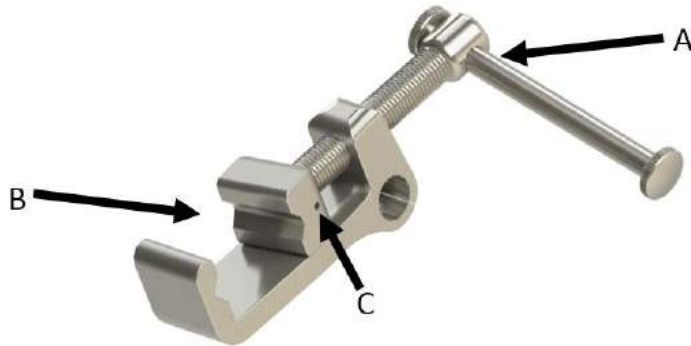
- The Greenberg® Retractor System is provided NON-STERILE. Remove all transport packaging and properly clean and sterilize the device prior to each use.
- For optimal security of components, place in a specialized tray such as the VersaPak™ Tray (part number 50-8535) for protection.
- Read, follow, and keep these instructions for use. Personnel using this retractor system should have the requisite training, knowledge, and experience. Use this retractor system only in accordance with its intended use.
- Prior to each use, inspect the retractor system for loose, bent, broken, cracked, worn, or fractured components.
- Do not use the retractor system if it is damaged or defective.
- Replace any damaged components immediately with original replacement parts, contact your local Symmetry Surgical representative or Symmetry Surgical Customer Service: Phone: 1-800-251-3000 or E-mail: customerservice@symmetrysurgical.com

Greenberg Simplicity® Components

The foundation of the original Greenberg® Retractor system consists of primary, secondary, floating secondary clamps, and short couplers. This system provides a modular approach to retraction and consists of many individual components. Users who prefer a less complex approach may opt to replace the original Greenberg® Retractor System primary, secondary, floating secondary clamps, and short couplers with the Greenberg Simplicity®. There are two Greenberg Simplicity® part numbers.


Greenberg Simplicity® Primary Clamp Catalog no. 50-1517

The Greenberg Simplicity® Primary Clamp is the base of the Greenberg Simplicity® assembly, which is an optional component of the Greenberg® Retractor System. The Greenberg Simplicity® Primary Clamp is attached to a skull clamp, horseshoe headrest, or bed frame rail by turning the vise handle (A) clockwise until the opening (B) is secured onto the base or arms of the skull clamp, horseshoe headrest, or bed frame rail. The primary clamp contains flush holes (C) on both sides of the rotating jaw to facilitate cleaning of this joint.

Greenberg Simplicity® Secondary Arc Catalog no. 50-1518

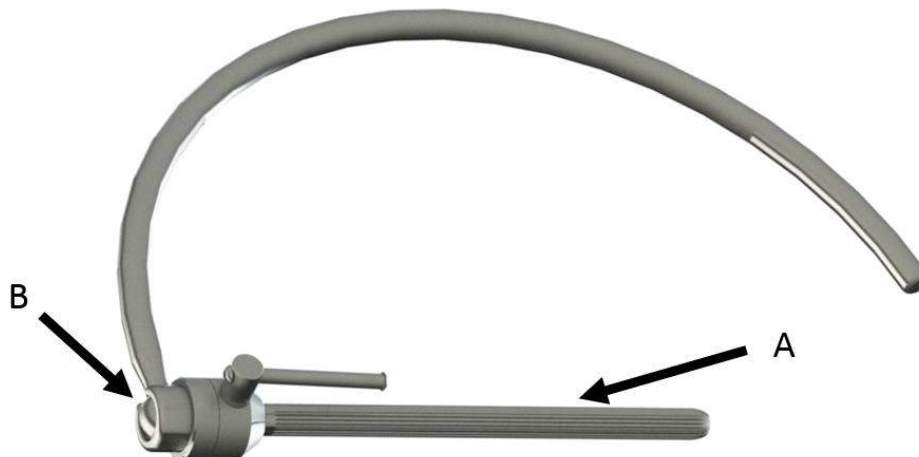The Greenberg Simplicity® Secondary Arc is a single piece framework on which retraction, hand rest(s), and accessory instruments are attached. The Simplicity Secondary Arc is secured once the serrated shaft (A) is placed through the Simplicity Primary Clamp and the clamp is fully tightened. The Simplicity Secondary Arc incorporates a universal joint (B) that enables conical adjustment of the bar's position.

Greenberg Simplicity® General Use

NOTE: Once the patient is properly positioned within a skull clamp, the patient is draped in the sterile field. The entire assembly process is carried out by sterile technique, with the Greenberg Simplicity® components being applied on top of the sterile drapes covering the skull clamp. The exact Greenberg® assembly configuration will vary based upon surgeon preference, patient position, and type of procedure.

Step 1—Determining the position of the Greenberg Simplicity® Primary Clamp

The primary clamp may be placed in several different orientations on the skull clamp depending upon the desired location of the secondary arc. It is recommended that the clamp be mounted such that the ribs of the skull clamp's supports are positioned within the horizontal grooves through the jaws of the primary clamp. The clamp may also be positioned around any two relatively parallel smooth surfaces located on the skull clamp. The jaws of the clamp must contact as much surface area of the skull clamp as possible. Ensure that the angles of both primary clamp jaws are tangent to their mounting surfaces on the skull clamp.

Step 2—Mounting the Greenberg Simplicity® Primary Clamp to the skull clamp (Figure 1 & Figure 2)

Position the primary clamp in the desired location and lightly tighten the clamp by turning the vise handle (A) clockwise until the clamp supports its own weight. While further securing the primary clamp with one hand, slide the spline of the secondary arc through the primary clamp in the desired orientation. Do not attempt to rotate the serrated shaft of the secondary arc once inserted into the primary clamp, as doing so could damage the serrated shaft.

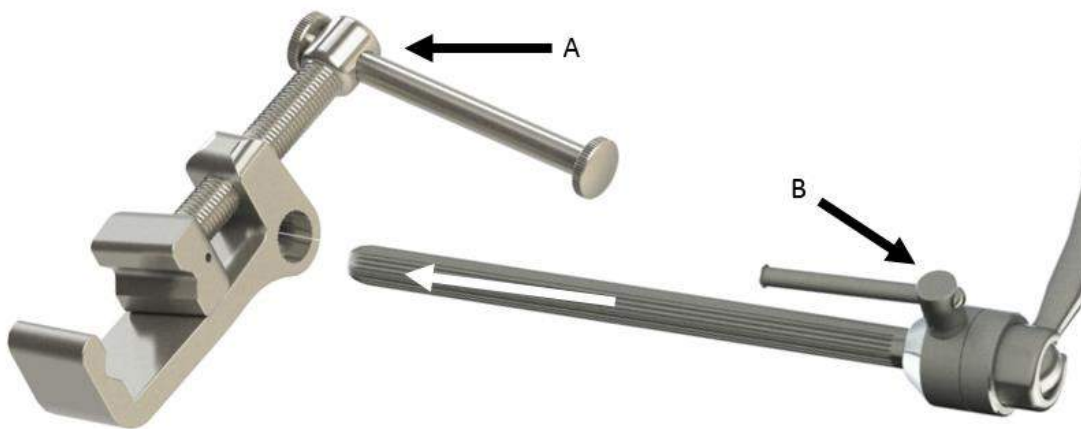
Figure 1: Inserting Secondary Arc

Once the secondary arc is inserted into the primary clamp to the desired amount, fully tighten the primary clamp vise handle (A) to secure the assembly. Turning the secondary arc universal joint adjustment lever (B) counterclockwise permits adjustment of the arc. To stabilize the arc in the desired position, turn the joint lever (B) clockwise until tightened. The slot in the collar of the universal joint permits 90° positioning of the secondary arc while maintaining stability.

Figure 2: Greenberg Simplicity® on Skull Clamp

NOTE: The secondary arc should be inserted into the primary clamp such that a portion of the secondary arc's serrated shaft protrudes from both sides of the clamp.

Step 3—Attaching Retractor Arms (Figure 3)

Retractor arms may be attached to the secondary arc. Greenberg® Tapered and Malleable Retractor Blades and other Greenberg® accessories can be mounted on the opposing ends of the retractor arms to enable 360° retraction and dissection of the brain.

Figure 3: Attaching Retractor Arms

Step 4—Attaching Micro Instrument Holders

Micro instrument holders may be attached to the secondary arc. Micro-suction and microdissection instruments can be mounted on the opposing end of the micro instrument holder.

Step 5—Attaching a Hand Rest (Figure4)

A hand rest may be mounted next, if desired, by affixing it to the serrated shaft of the secondary arc.

Figure 4: Attaching Hand Rest

Care and Maintenance

All Greenberg® instruments should be disassembled from one another prior to cleaning.

Cleaning, Sterilization, and Lubrication

Clean, sterilize, and lubricate all Greenberg® instruments before first use and after each use according to the recommended directions in Symmetry Surgical IFU- 204233.

Ordering Information

Catalog Number	Description
50-1517	Greenberg Simplicity® Primary Clamp
50-1518	Greenberg Simplicity® Secondary Arc

Warranty

Symmetry Surgical warrants that this medical device is free from defects in both materials and workmanship for one (1) year from the date of purchase. Any other express or implied warranties, including warranties of merchantability or fitness, are hereby disclaimed. Suitability for use of this medical device for any particular surgical procedure should be determined by the user in conformance with the manufacturer's instructions for use. There are no warranties that extend beyond the description on the face hereof. **Abuse or misuse of the product or failure to comply with the instructions for use shall void this warranty.**

Greenberg® is a registered trademark of Louise Greenberg.

Service and Repair

For service or repair outside the United States, contact your local Symmetry Surgical representative. Always include the purchase order number and a written description of the problem.

Inside the United States, send instruments for service or repair to: Symmetry Surgical
3034 Owen Drive, Antioch, TN 37013 USA
Symmetry Surgical Customer Service: Phone: 1-800-251-3000 or E-mail:
customerservice@symmetrysurgical.com

LABELING SYMBOLS

Symbol for Manufacturer; the name and address of the Manufacturer is next to this symbol

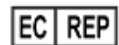
Caution-Consult accompanying documents

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