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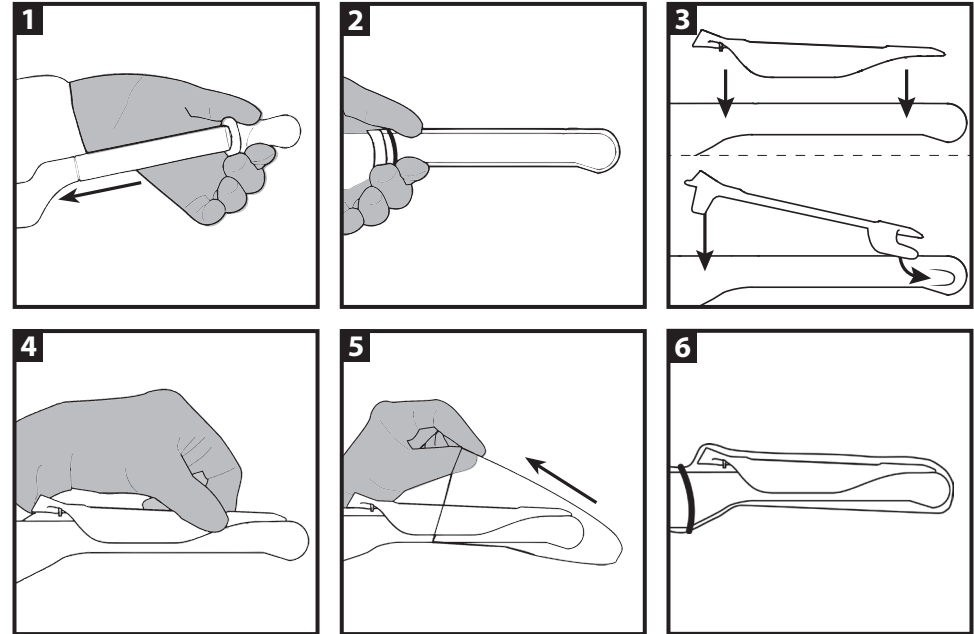
 Aspen Surgical Products, Inc.  
 6945 Southbelt Dr. SE, Caledonia, MI 49316  
 phone +1 616.698.7100 toll-free +1 888.364.7004  
 aspensurgical.com

 Emergo Europe  
 Prinsessegracht 20  
 2514 AP The Hague  
 The Netherlands

Made in Mexico

1-041-9000WEB | Rev 15 | 30JAN2022

## ENGLISH

## Instructions For Use

for Disposable Endocavity Needle Guide  
ULTRASOUND ACCESSORIES

Prepare probe per system's operational manual.

Always verify needle follows guidelines prior to use. If needle guide does not fit properly, do not use. Use appropriate needle length to reach target area.

1. Place gel inside cover and/or on probe scanning surface. Place probe cover over probe.
2. Secure cover with bands.
3. Place needle guide onto probe and snap into position.
4. Make sure needle guide is securely attached to probe.
5. Place second probe cover over probe/needle guide.
6. Secure cover with bands.

This needle guide is single-use only. **Do not reuse. Do not resterilize. Reuse, reprocessing or re-sterilization of the single use devices may create a risk of cross contamination, patient infection, and / or malfunction of the device.**

After use, dispose of single-use components as infectious waste.  
 Do not use if sterile package is damaged.

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