CLASSIC PLUS[®] Scissors with STELLITE[™] Inserts

Symmetry Surgical Inc.
3034 Owen Drive
Antioch, TN 37013 USA
☎ 1-800-251-3000
Fax: 1-615-964-5566
www.symmetrysurgical.com

Symmetry Surgical GmbH
Maybachstraße 10
78532 Tuttlingen, Germany

☎ + 49 7461 96490
Fax: + 49 7461 77921

LCN 360-5000-85/L

© 1999–2015 Symmetry Surgical

♻ Revised 10/15

ENGLISH

IMPORTANT INFORMATION

Please Read Before Use

CLASSIC PLUS® Scissors with STELLITE™ Inserts

Caring for Scissors with STELLITE™ Inserts

STELLITE™ inserts are used in the jaws of scissors because they maintain a sharp edge. In certain delicate patterns, however, it is not possible to fully reinforce the STELLITE™ tip of the scissors. Because of this, the STELLITE™ insert is susceptible to breakage at the tip (see Figure 1). It is important to protect this STELLITE™ insert when handling these scissors and to process the instrument with caution, particularly during transit, sterilization, and cleaning.

WARNING

To prevent breakage of the blades at the tip when cutting, do not twist or overload the blades of STELLITE™ insert scissors.

Reprocessing

Refer to Symmetry Surgical Instructions for Use 204233 for updated information regarding reprocessing the instrument.

Copies of this instruction sheet are available by requesting IFU 204233 from Symmetry Surgical Customer Service at:

Telephone: 800 251 3000

Fax: +1 615 964 5566

E-mail: customerservice@symmetrysurgical.com

Service and Repair

For service or repair outside the United States, contact your local Symmetry Surgical representative.

Inside the United States, send instruments for service or repair to:

Symmetry Surgical Inc.
3034 Owen Drive,
Antioch, TN 37013 USA

Warranty

Symmetry Surgical warrants that this medical device is free from defects in both materials and workmanship. **Any other express or implied warranties, including warranties of merchantability or fitness, are hereby disclaimed. Suitability for use of this medical device for any particular surgical procedure should be determined by the user in conformance with the manufacturer's instructions for use. There are no warranties that extend beyond the description on the face hereof.**

CLASSIC PLUS® is a registered trademark of Symmetry Surgical

STELLITE™ is a trademark of Cabot Corporation

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Téléphone : 800 251 3000

Fax : +1 615 964 5566

E-mail : customerservice@symmetrysurgical.com

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