

Instructions for Use: REF# SS50 (single set) and
SS100 (double set)

1. Remove from packaging and place each strap at the desired location on opposite sides the O.R. Table Rail.
2. Slide each end of strap between the O.R. Table and the table rail, and bring the end of the strap through the plastic ring at the opposite end of the strap. Pull the strap completely through the ring until the strap is completely secured to the table rail.
3. When positioning the strap with the Velcro "hook" material, ensure that the hook material is facing the plastic ring to ensure smooth passage of the strap through the plastic ring.
4. Place each section of the strap across the patient at the desired position and ensure that both strap sections are securely connected.
5. Discard after use.



Not made with natural rubber latex



Manufactured For:


PRIME
MEDICAL
1410 E Weisgarber
Knoxville, TN 37909
865-776-8994



QNET BV
Kantstraat 19
NL-5076NP Haaren
The Netherlands