

## Instructions for use

### Berkette Beach Chair Limb Positioning Kit 711200

Previously Beatty® reorder # 100801

#### Intended Use:

Intended to support and maintain the position of an anesthetized patient during surgery.

#### Intended User:

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

#### Precautions:

- Used with ARTHREX® TRIMANO® Shoulder Positioner.
- Ensure wrap is securely attached to the positioning device while positioning arm.
- Ensure wrap is properly secured to the arm to avoid possible contamination of the sterile field.
- Do not overtighten self-adhering wrap and restrict blood flow.
- This device is single use only.
- Do not re-sterilize.
- Do not use if product is damaged or particulate is found in sterile packaging.

#### Disposal:

Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

#### Instructions for Use:

1. Present sterile wrapped goods to sterile field following appropriate sterile technique.
2. Unwrap sterile CSR wrap.
3. Place foam grip in patient hand before wrapping.
4. Wrap foam arm holder securely around arm after it has been aseptically prepped for surgery.
5. Wrap device with self-adhering wrap after application.
6. Place foam arm holder securely in the positioning device.
7. Discard after use.

Arthrex® and Trimano® are registered trademarks or trademarks of Arthrex®, Inc. ("Arthrex"). Aspen Surgical Products, Inc. is not affiliated with, endorsed by, or sponsored by Arthrex®.



Aspen Surgical Products, Inc.

6945 Southbelt Dr. SE, Caledonia, MI 49316

phone +1 616.698.7100 toll-free +1 888.364.7004

[aspensurgical.com](http://aspensurgical.com)



STERILE | R

**Rx ONLY**