INSTRUCTIONS FOR USE: Scope HelpOR™ EN  
Scope Clearing / Cleaning Device

## INDICATIONS FOR USE / CLINICAL BENEFITS

The Scope HelpOR is indicated to be used before and during endoscopic and laparoscopic surgery to prevent condensation and fogging of the scope, and keep the scope lens and trocar valves free of debris.

## CONTRAINDICATIONS

This device is not intended for use except as indicated.

## CAUTIONS / WARNINGS

- Device is sterile if the package is dry, unopened and undamaged. Do not use if the pouch is damaged or the packaged seal is broken.
- Verify compatibility. Inspect the endoscope / laparoscope and trocar for sharp edges that could interfere with the components of the Scope HelpOR.
- Device is designed for short term use. Do not use for more than 24 hours.
- Peak temperature of the Lap Scope WarmOR (approximate): 50 +/- 5°C.
- Lap Scope WarmOR is designed to be used with 6V AC adaptor (SW-PA-6V) ONLY. The 6V AC adaptor must be approved for medical use. NOTE: The adaptor is reusable and non-sterile.
- Improper use of the Scope HelpOR may lead to patient injuries and/or damage of the endoscope / laparoscope and trocar.
- DO NOT place Lap Scope WarmOR on patient.
- The Scope HelpOR was designed, tested and manufactured for single patient use only. Reuse or reprocessing of this device may lead to its failure and subsequent patient injury.
- DO NOT re-sterilize the device. Re-sterilization may compromise the integrity of the device and create the risk of contamination and unintended patient infection.
- Upon completion of the procedure, the Scope HelpOR should be discarded in biohazardous waste as per standard procedure.

## INSTRUCTIONS FOR USE

Open sterile pouch, remove the Scope HelpOR and place onto sterile field.

## Using the Lap Scope WarmOR

- Connect reusable, non-sterile 6V AC adaptor (SW-PA-6V) into power socket.
- Pass Lap Scope WarmOR cord to the scrub nurse/theatre technician to connect Lap Scope WarmOR to 6V AC adaptor (Figure 1).
- Insert distal tip of endoscope/laparoscope into the Lap Scope WarmOR and allow to warm for 5-10 minutes (for optimal performance).
- Carefully remove endoscope/laparoscope from the Lap Scope WarmOR ready for use.

## Using the Scope CleanOR (microfiber cloth and two trocar swabs)

- Anytime the scope is removed from the body:
- Clean the scope lens by using the microfiber cloth.
  - To remove debris from the trocar valves use the small trocar swab for 5-8mm trocars, and the larger trocar swab for 10-12mm trocars (Figure 2). Note: Trocar cannula must be fully inserted into the abdominal cavity prior to use of the Trocar CleanOR.
  - Select the appropriate trocar swab and insert its foam head into the trocar valve. Advance the swab into the valve, but do not extend the foam head beyond the distal end of the trocar. Move the swab back and forth to absorb any fluid and clean debris from the cannula. Do not release the trocar swab. Note: The shafts of the trocar swab are radiopaque. Trocar CleanORs must be used under direct vision.
  - Check the cannula for free passage. If this is not the case, repeat the process.
  - Before disposing of the trocar swabs and microfiber cloth visually inspect the devices to ensure they are intact and all components have been retrieved.

Any serious incident that has occurred in relation to this device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Figure 1

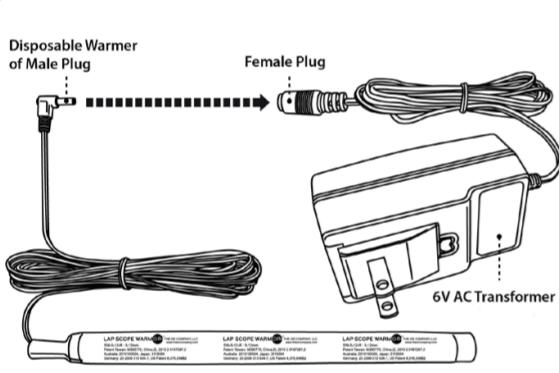
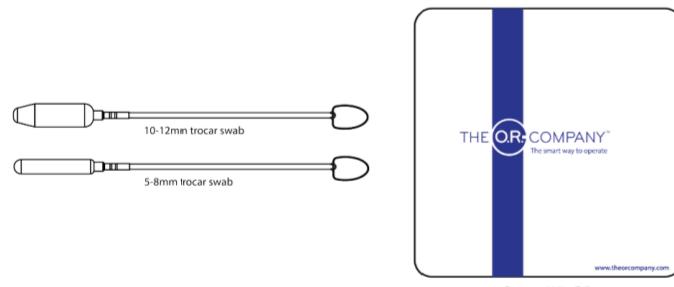
Figure 2

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