INSTRUCTIONS FOR USE

Bard Parker®

aspensurgical.

Conventional Blade System

EC REP

Emergo Europe
Westervortseidijk 60
6827 AT Arnhem
The Netherlands

CE
2797

Aspen Surgical Products, Inc.
6945 Southbelt Drive SE
Caledonia, MI 49316 USA
Phone +1 616-698-7100
Toll-Free 888-364-7004
aspensurgical.com

For Country of Origin See Product Label

English

Bard-Parker® Conventional Blade System

Intended Use

Intended for tissue separation and other procedures that require a sharp blade to puncture or cut.

Intended User/ Patient Target Groups

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

Instructions for Use:

- The way in which a surgical blade is handled prior to use can affect the way in which the blade performs during use. Take care to ensure the cutting edge of a blade does not become damaged once removed from its packaging. This includes dropping the blade into a metal bowl or other container that could potentially dull the blade. It also includes not gripping the blade with a forceps or needle clamp across the cutting edge of the blade.
- Keep in mind that surgical blades are sharp instruments. Take proper precautions in handling the blade so as not to injure yourself or others before, during, or after use. Seek proper training and instruction on use before handling.
- During use avoid twisting, bending or putting excessive force or strain on the blade in order to help prevent breakage.
- A singular blade is intended to cut tissue up to 24 inches.
- If blade becomes dull or breaks, dispose and replace product.
- Do not use product if product is damaged.

Attaching the Surgical Blade

- Peel pouch open. Grip blade with forceps or needle clamp and remove from pouch, taking special care not to grip across the cutting edge of the blade itself.
- Holding on to the thickest part of blade with forceps or needle clamp, insert the handle into blade track.
- Slide blade back onto handle until it clicks into position.

Removing the Surgical Blade

- Grip bottom of blade with forceps or needle clamp and pull up to loosen blade from handle.
- Push blade forward from handle to remove blade, being careful not to jerk the blade off the track.

Warning

Use of device could result in infection/contamination and/or device failure which could lead to patient harm.

Disposal

Dispose in approved puncture-resistant sharps container according to facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

Notice to Users and/or Patients in EU

Any serious incident that has occurred in relation to the device, should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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AB'deki Kullanıcılara ve/veya Hastalara Yönelik Bildirim
Emergo Consulting (UK) Limited