disposable

Ready-Rolled® FootPrinter

How to use the Disposable FootPrinter with Nontoxic Ready-Rolled® Ink

Necessary Materials:

Disposable Ready-Rolled® FootPrinter, Newborn Identification Form or Heirloom Birth Certificates, soap, water, gauze pads and clipboard.

Disposable FootPrinter, Newborn Identification Form or Heirloom Birth Certificates, soap, water, gauze pads and clipboard.

Taking the Infant's Foot Prints:

- 1) Remove one FootPrinter from the dispenser box and attach the form to the clipboard.
- 2) Cleanse the infant's feet with soap and water. Remove vernix and cellular debris. Rinse and dry thoroughly.
- 3) Hold the infant's ankle between your thumb and middle finger and place your index finger behind the infant's toes. Place the infant's foot gently on the inked surface of the FootPrinter. The infant's knee should be flexed and close to the infant's body.
- 4) Gently press the form against the infant's inked foot. Do not roll the foot back and forth.

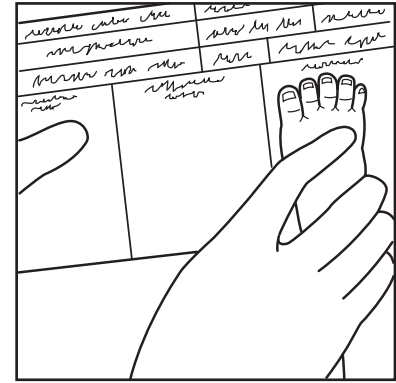
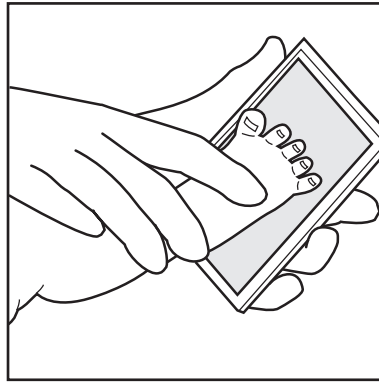
Once the foot imprints are taken, the infant's feet should be cleansed with soap and water to remove the excess ink. Do not use alcohol or strong cleansers, which may burn or dry the infant's skin.

Taking the Mother's Finger Imprints:

Ensure the mother's right hand is clean and dry. Using the same FootPrinter, take the mother's right index finger and gently cover it with ink. Using a single left to right motion, gently roll the mother's inked finger on the paper. Do not roll the finger back and forth.

Note: If ridges in the ball of the foot and heel are not clear or if the print smears due to too much pressure being applied, immediately make another imprint.

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