

## Instructions for use

### Marco Shoulder Stabilization Kit 711100

Previously Beatty® reorder # 101601

#### Intended Use:

Intended to support and maintain the position of an anesthetized patient during surgery.

#### Intended User:

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

#### Precautions:

- For use with Smith & Nephew® SPIDER™ Limb Positioner 7210569.
- Ensure wrap is securely attached to the positioning device while positioning arm.
- Ensure wrap is properly secured to the arm to avoid possible contamination of the sterile field.
- Do not overtighten wrap and restrict blood flow.
- This device is single use only.
- Do not re-sterilize.
- Do not use if product is damaged or particulate is found in sterile packaging.

#### Disposal:

Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

#### Instructions for Use:

1. Place foam grip in patient hand before wrapping.
2. Wrap foam arm holder securely around arm after it has been aseptically prepped for surgery.
3. Place foam arm holder securely in the positioning device.
4. Discard after use.

Smith & Nephew® SPIDER™, is a registered trademark of Smith & Nephew PLC ("Smith & Nephew"). Aspen Surgical Products, Inc. is not affiliated with, endorsed by, or sponsored by Smith & Nephew.



Aspen Surgical Products, Inc.

6945 Southbelt Dr. SE, Caledonia, MI 49316

phone +1 616.698.7100 toll-free +1 888.364.7004

[aspensurgical.com](http://aspensurgical.com)



STERILE | R

**R<sub>x</sub> ONLY**