Bovie® SharkSkin Pencil Adapter
Electrosurgical Pencil Adapter

EN

REF SS95, SS95-1

QTY 20/Case

10 ft (3.05 m) tubing

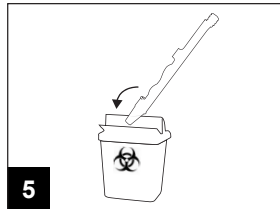
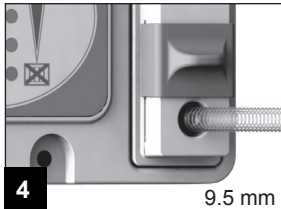
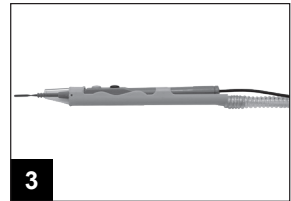
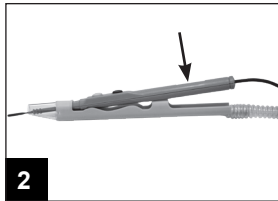
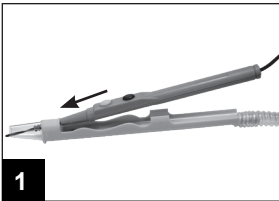
Indications For Use: Bovie® SharkSkin Pencil Adapter is an aspiration sheath that fits over an electrosurgical pencil body and leaves the tip or blade exposed. This device is considered an accessory to an electrosurgical unit (ESU). Bovie® SharkSkin Pencil Adapter removes surgical smoke during surgical procedures that uses an ESU for cutting and cauterizing tissue. This device is used in conjunction with a suction (vacuum) source.

Contraindication: This device is not to be used for microsurgery.

Precautions: For single use only. Do not reuse, reprocess, or re-sterilize. Reuse, reprocessing, or re-sterilization will compromise integrity of Bovie® SharkSkin Pencil Adapter and result in patient injury, illness, or death.

Instructions for Use: Using aseptic technique, open packaging and remove Bovie® SharkSkin Pencil Adapter onto the sterile field. Follow steps below.

Report complaints involving this device to complaint@symmetrysurgical.com or to your local distributor.

STERILE EO Rx ONLY

MD

<http://eifu.commed.com>

AU SPONSOR:
The OR Company Pty Ltd
1/32 Silkwood Rise
Carrum Downs
VIC 3201 Australia

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European Commission requires that any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Bovie®

Symmetry Surgical, Inc.
3034 Owen Drive, Antioch, TN 37013, USA
USA & Canada: 800.251.3000 Fax: 800.342.3272
Email: customerservice@symmetrysurgical.com
For International Inquiries: +1 615.964.5532
Email: internationalorders@symmetrysurgical.com

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Bovie®

 Symmetry Surgical, Inc.
3034 Owen Drive, Antioch, TN 37013, USA
USA & Canada: 800.251.3000 Fax: 800.342.3272
E-mail: customerservice@symmetrysurgical.com
For International Inquiries: +1 615.964.5532
Email: internationalorders@symmetrysurgical.com
905293 Rev D 12/2021

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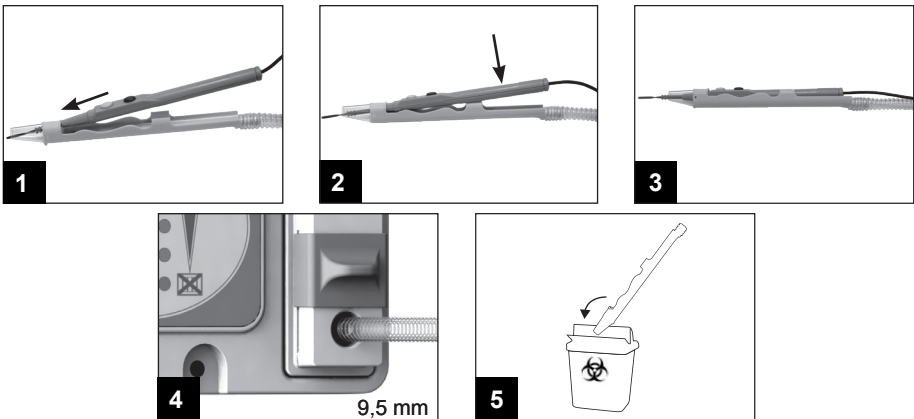
Bovie®
Symmetry Surgical, Inc.
3034 Owen Drive, Antioch, TN 37013, USA
USA & Canada: 800.251.3000 Fax: 800.342.3272
Email: customerservice@symmetrysurgical.com
For International Inquiries: +1 615.964.5532
Email: internationalorders@symmetrysurgical.com
905293 Rev D 12/2021

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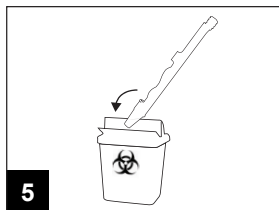
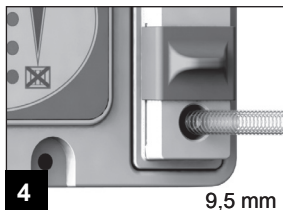
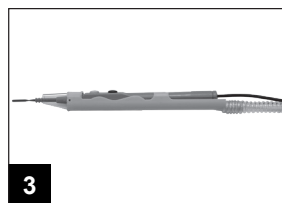
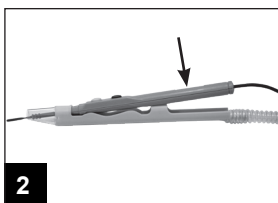
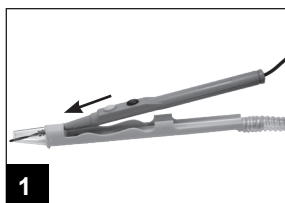
Bovie

Symmetry Surgical, Inc.
3034 Owen Drive, Antioch, TN 37013, USA
USA & Canada: 800.251.3000 Fax: 800.342.3272
Email: customerservice@symmetrysurgical.com
For International Inquiries: +1 615.964.5532
Email: internationalorders@symmetrysurgical.com
905293 Rev D 12/2021

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MDSS GmbH
CONMED Corporation