For product information, contact: Yderligere produktoplysninger fås ved at kontakte: Næm voor productinformatie contact op met: Pour obtenir des informations sur le produit, veuillez contacter : Für Produktinformation kontaktieren Sie: Για πληροφορίες σχετικά με το προϊόν, επικοινωνήστε με: Per informazioni sul prodotto, contattate: 製品情報、お問い合わせ: Para obter informações sobre o produto, contacte: Para obtener información del producto, póngase en contacto con: För produktinformaton, kontakta:
3230 Commerce Center Place Louisville, Kentucky, 40211 (502) 813-5022 ⇨ (800) 297-6344 See our products at Se vores produkter på Ga voor onze producten naar Découvrez nos produits sur Siehe unsere Produkte auf Δείτε τα προϊόντα μας στην ιστοσελίδα. È possibile vedere i nostri prodotti all'indirizzo web 当社の製品は以下でご覧になれます Consulte os nossos produtos em Consulte nuestros productos en Se våra produkter på
Made in the U.S.A. Fremstillet i U.S.A. Vervaardigd in de VS Fabriqué aux États-Unis In den USA hergestellt Κατασκευασμένο στις ΗΠΑ Prodotto negli USA 作成・印刷(米国) Fabricado nos EUA Fabricado en EE.UU. Tillverkad i USA

RY ONLY Reusable Electrological Devices Genanvendelige elektrokirurgiske instrumenter Herbruikbare elektrochirurgische instrumenten Appareils d'électrochirurgie réutilisables Wiederverwendbare elektrochirurgische Geräte Επαναχρησιμοποιήσιμες Ηλεκτροχειρουργικές Συσκευές Dispositivi elettrochirurgici riutilizzabili 再使用可能電気外科装置 Dispositivos electroquirúrgicos reutilizáveis Dispositivos electroquirúrgicos reutilizables Elektrokirurgiska enheter för flergångsbruk

ENGLISH

Description:

Olsen Medical® reusable electrological devices are sold as non-sterile unless otherwise noted. Each reusable device is designed to withstand a minimum of 20 sterilization cycles when properly cared for and sterilized according to the instructions below. These devices are designed to function as active electrodes or as accessories to active electrodes. This family of devices includes, but is not limited to: monopolar and bipolar forceps, pens/pencils and electrode handles, electrodes, cords and adapters.

Indications:

Monopolar Cord and Adapters - Olsen Medical® electrological cords and adapters are designed to bring high frequency electrical power from the electrological generator to the active electrode and are intended for use with a maximum voltage of 3500 Volts. Bipolar Forceps and Cords - Olsen Medical® Bipolar Forceps and cords are intended for use in coagulating tissue during surgical procedures and are intended for use with a maximum voltage of 500 Volts. Monopolar Forceps - Olsen Medical® Monopolar Forceps are intended for use in cutting and coagulating tissue during surgical procedures and are intended for use with a maximum voltage of 2000 Volts. Lietz Electrodes - Olsen Medical® Lietz Electrodes are intended for use in coagulating and cutting tissue during surgical procedures and are intended for use with a maximum voltage of 2000 Volts. Monopolar Electrodes - Olsen Medical® Electrodes are intended for use in coagulating and cutting tissue during surgical procedures and are intended for use with a maximum voltage of 3500 Volts. Pens/Pencils and Handles - Olsen Medical® pens/pencils and handles are designed to hold active electrodes and are intended for use with a maximum voltage of 3000 Volts.

Handling:

- Remove and discard the tip protector on all devices before use.
- Inspect device before each use. Discard or send for repair if damaged.
- Device use and application to be determined by physician.
- Cleaning and reprocessing steps: reference Symmetry Surgical Instructions for Use (IFU) #204233 for updated information regarding reprocessing the instrument.
 - IFU # 204233 is available at www.Symmetryurgical.com/IFU

Storage:

Reusable devices must be stored individually in a protective container in a clean, dry, moisture free area with suitable protection from extreme temperature and humidity.

Sterility:

- These devices are sold as non-sterile and must be steam sterilized prior to each use. Other methods of sterilization are not recommended and their use will void the warranty.
- Instruments designed to come apart must be disassembled prior to cleaning.
- Arrange all devices to allow access of steam to all surfaces. Open hinged devices and ensure devices are disassembled if it is recommended.

Steam Sterilization:

Sterilization Cycle	Temperature	Time	Minimum Dry Time	Device Configuration
Prevacuum Flash	270°F (132°C)	3 Minutes	–	Sterilization Tray
Prevacuum	270°F (132°C)	4 Minutes	20 Minutes	Wrapped/ Individually Pouched
Gravity Displacement	270°F (132°C)	10 Minutes	10 Minutes	Wrapped

Caution:

- Inspect every device before each use. Damaged electrological devices may cause electrical burns to the patient or physician.
- Active electrological devices must be placed away from the patient when not in use. A holster is recommended.
- Use a wet sponge to remove eschar build up from the forceps tips.
- Consult RF generator Owner’s Manual. Use only in the active RF generator output connections.
- Do not perform electrologicals in the presence of flammable anesthetics or other flammable gases, liquids or objects, or in the presence of oxidizing agents, as fire could result.
- Avoid unnecessary or prolonged activation. Activate only when touching or immediately next to the intended point of contact.
- Non-insulated switching contacts of hand controlled forceps must not come in close proximity to or touch the skin or bodily tissue to prevent a burn to the patient or physician.
- Electrode must be compatible with the pen/pencil or handle and ensure that the insulated sleeve fits securely inside the handle. Improperly sealed electrode may cause burn or injury.
- Do not bend, cut, or modify any electrological device before or during surgery.
- Keep the voltage/power settings at the lowest possible level to achieve the desired effect.
- Establish a protocol for device examination and maintenance.

Warning:

All products are warranted to be free from defects in workmanship and materials when used for its intended purpose. Any product that proves defective in workmanship or materials will be repaired or replaced at our discretion free of charge. Repair, modification, or alteration to any product that is performed by anyone not authorized by Olsen Medical® will result in the immediate loss of warranty.

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OLSEN® is a registered trademark of Olsen Medical.