EN REUSABLE BIPOLAR FORCEPS CORD

Store in a cool, dry place.	Device is rated for 1250 Vpeak
WARNING:	DO NOT USE IN THE PRESENCE OF FLAMMABLE MATERIALS. DO NOT PLACE ACTIVE ACCESSORIES NEAR OR IN CONTACT WITH FLAMMABLE MATERIAL (GAUZE, SURGICAL DRAPES). FIRE COULD RESULT.
WARNING:	ACCESSORIES AND FORCEPS MUST BE PLACED IN A CLEAN, DRY, NONCONDUCTIVE AND HIGHLY VISIBLE AREA WHEN NOT IN USE.
WARNING:	DO NOT USE A CORD WITH VISIBLE INSULATOR DAMAGE. USING A DAMAGED CORD MAY RESULT IN BURNS TO THE USER OR PATIENT.
CAUTION:	This product is sold NON-STERILE. Sterilize prior to use..

Instructions for Use:

- Examine the cord for cracks, tears, pinholes, or other damage. Connect the cord plug into the bipolar receptacle of the generator.
- Follow all instructions for use that are provided with the forceps in use. Attach forceps to the cord ensuring that the forceps pins are fully seated in the cord receptacle. Position the cord so that it does not contact the patient or other electrical cables. Using the lowest power setting, test the forceps and cord by pressing the footswitch. If the generator fails to activate, check the forceps connection with the cord and secure if necessary.

Cleaning Instructions:

- Remove all gross matter (blood, tissue) by wiping the cord with a gauze pad or soft cloth dampened with warm tap water until all visible debris is removed.
- Soak the cord in an enzymatic detergent mixture diluted according to the manufacturer’s instructions. The cord may also be cleaned using a soft bristled cleaning brush to remove visible debris.
- IMPORTANT: Abrasive material should not be used in the area of the barcode when cleaning the device.
- Remove cleaning agents from the cord by wiping with a cloth dampened with tap water.
- Allow the cord to dry thoroughly prior to sterilization.

Sterilization Instructions:


NUMBER OF USES WHEN STERILIZED BY STEAM AUTOCLAVE: 25.

- Place the cord in an autoclavable pouch or wrapped cloth (consult the AAMI recommended practices for steam sterilization). Steam autoclave at 250°F. (121°C. gravity displacement) for 30 minutes.
- Allow the cord to cool to room temperature prior to use.
- DO NOT USE “FLASH” AUTOCLAVE CYCLES. THE CORD IS NOT COMPATIBLE WITH THE HIGHER TEMPERATURES USED IN THESE CYCLES.

REUSABLE BIPOLAR FORCEPS CORD

REF A827BP, A827F, A827V

R_x ONLY

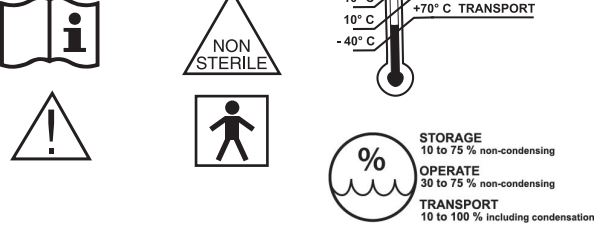Bovie®

Bovie Medical Corporation
5115 Ulmerton Road
Clearwater, Florida 33760 USA

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