FOOTSWITCH INSTRUCTIONS FOR USE EN

RF- 1253B, BV- 1254B

Indications for Use

Footswitch BV-1253B and Footswitch BV-1254B are indicated for tissue incisions or when bleeding control is desired.

Contraindications

The use of Footswitch BV-1253B and Footswitch BV-1254B is contraindicated in the presence of flammable anesthetics, oxygen-enriched or explosive atmospheres.

Intended Use

Footswitch BV-1253B and Footswitch BV-1254B are used to activate the cut and / or coagulation functions of an electrosurgical generator when using an attached Foot-switchable handle/pencil or other accessory.

Clinical Benefits

The primary patient benefits are to facilitate a surgical procedure through providing a means for cutting, coagulation, and ablation of tissue.

Intended Users

- Education – Trained physician, physician's assistant, nurse, nurse practitioner. No maximum.
- Knowledge – no maximum.
- Minimum:

- Understands electrosurgery and electrosurgical techniques;
- Read and understands supplied User's Guide (Accompanying Document);
- Understands hygiene.
- Maximum: There is no maximum.

- Experience:
- Minimum:
- Some training on techniques or training under surveillance/supervision;
- No special experience needed.
- Maximum: There is no maximum.
- Permissible Impairments:
- Mild reading / vision impairment or vision correction to 20/20.
- Hearing impaired by 40% resulting in 60% of normal hearing at 500 Hz to 2.0 kHz

Patient Population

- Patient should not be user
- Age: Infant to geriatric
- Weight: No restriction

Device Description

The BV-1253B Footswitch is a dual pedal, non-sterile, reusable HF accessory intended to to activate the Monopolar RF output on the A1250S, A3250, A3350, A2250, A2350, IDS-210 and IDS-310 Bowie generators. The footswitch is comprised of two ped- als, a cable and connector.

The BV-1254B Footswitch is a single pedal, non-sterile, reusable HF accessory intended to to activate the Bipolar RF output on the A3250, A3350, A2250, A2350, IDS-210 and IDS-310 Bowie generators. The footswitch is comprised of a single pedal, a cable and connector.

Activating the Footswitch

- Follow the generator instructions for use to set up the system for surgery.
- Connect the footswitch connector to the Footswitch receptacle on the generator. Note: the connector is designed to be attached by screwing the connector onto the corresponding socket on the generator.
- To activate the generator, depress the foot pedal of the footswitch. On the BV-1253B the yellow pedal actives the yellow monopolar modes. The blue pedal activates the blue monopolar modes. On the BV-1254B the blue pedal activates the bipolar modes.
- To halt activation, release the foot pedal of the footswitch.

Notices:

- Electrosurgical equipment and accessories are intended to be used by health professionals educated in their use.
- After Use or upon determining product can no longer be used due to wear or damage, safely discard in accordance with established procedures for biohazardous waste.
- Any serious incident that has occurred in relation to the device should be reported to Aspen Surgical Products Inc., the governing body, and the Competent authority of the member state in which the incident occurred.
- Please refer to the manufacturer of the generator for warnings, precautions, contraindications, undesirable side effects, measures to be taken, and limitations of use for the electrosurgical system and accessories.
- For applicable IFU/additional information: Visit https://www.aspensurgical.com/Resources/Documents/IFUs

Reusable Device:

Reusable devices are provided non-sterile and must be processed prior to use to include first use. Processing is defined on specific instructions provided with each device.

Cautions:

- The BV-1253B is only designed to be compatible with the A1250S, A3250, A3350, A2250, A2350, IDS-210 and IDS-310 Bowie generators. Do not use with any other generators or equipment.
- The BV-1254B is only designed to be compatible with the A3250, A3350, A2250, A2350, IDS-210 and IDS-310 Bowie generators. Do not use with any other generators or equipment.
- Do not drop Footswitchable parts.
- Before use, examine the electrosurgical unit and accessories for defects. Do not use cables or accessories with damaged (cracked, burned, or taped) insulation or connectors. No or unintended activation may occur.

Warnings:


- Do not use in the presence of flammable materials (e.g. alcohol, flammable anesthetics).
- No modification of this device is allowed.
- Do not immerse the Footswitch in liquids.
- Do not sterilize Footswitch.
- Inspect the packaging and the device for overall condition and physical integrity. Do not use the device if a damage or defect in packaging or/and device is noted.
- Use only with specific medical devices and accessories identified in the IFU. Use of this device with non-identified devices and accessories may cause unnecessary harm to the user and/or patient.
- Due to concerns about the carcinogenic and infectious potential of electrosurgical byproducts (such as tissue smoke plume and aerosols which may include toxic gases and vapors, live and dead cellular material, and viruses), protective eyewear, filtration masks, and effective smoke evacuation equipment should be used in both open and laparoscopic procedures. Contact Aspen Surgical Products, Inc. at customerservice@aspensurgical.com for additional information or to inquire about our smoke evacuation solutions.

Cleaning:

- Footswitch may be cleaned using a soft cloth moistened with a solution of water and mild detergent or disinfectant. Do not immerse in liquids.

Storage:

- Store in a cool, dry place.
- Avoid prolonged exposure to extreme temperatures.
- The environmental conditions for operation, storage and transport are listed below.

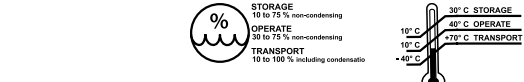Expected Service Life:

The expected service life of the BV-1253B and BV-1254B footswitches is 7 years.

Warranty:

- Aspen Surgical Products Inc., warrants each product manufactured by it to be free from defects in material and workmanship under normal use and service for the period(s) set forth below.
- Aspen Surgical Products Inc.'s obligation under this warranty is limited to the repair or replacement, at its sole option, of any product, or part thereof, which has been returned to it or its Distributor within the applicable time periods known below after delivery of the product to the original purchaser, and which examination discloses, to Aspen Surgical Products Inc.'s satisfaction, that the product is indeed, defective.
- This warranty does not apply to any product, or part thereof, which has been repaired or altered outside Aspen Surgical Products Inc.'s factory in a way so as, in Aspen Surgical Products Inc.'s judgment, to affect its stability or reliability, or which has been subjected to misuse, neglect, or accident.
- The warranty periods for Aspen Surgical Products Inc., is One year from date of shipment.

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