

Instructions for use

Erin Beach Chair Face Mask Replacement Strap 711002

Previously Beatty reorder # 061201

Intended Use:

Intended to support and maintain the position of an anesthetized patient during surgery.

Intended User:

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

Precautions:

- Replacement straps for the SMITH & NEPHEW® T-MAX™ Beach Chair 7210996.
- Disinfecting of straps between use should be performed per facility protocol
- Verify securement of straps to device before using.
- Do not use if straps are visually damaged or degraded.
- Do not sterilize.

Disposal:

Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

Instructions for Use:

1. Remove existing straps.
2. Replace straps.
3. Ensure straps are securely set in place.

Smith & Nephew® and T-MAX™ are registered trademarks or trademarks of Smith & Nephew PLC ("Smith & Nephew"). Aspen Surgical Products, Inc. is not affiliated with, endorsed by, or sponsored by Smith & Nephew.



Aspen Surgical Products, Inc.

6945 Southbelt Dr. SE, Caledonia, MI 49316

phone +1 616.698.7100 toll-free +1 888.364.7004

aspensurgical.com



R_x ONLY