Symmetry Surgical® OPTI-LENGTH® Spinal Instruments Symmetry Surgical® Spinal Instruments

Symmetry Surgical Inc.
3034 Owen Drive
Antioch, TN 37013 USA
☎ 1-800-251-3000
Fax: 1-615-964-5566
www.symmetrysurgical.com

Symmetry Surgical GmbH
Maybachstraße 10
78532 Tuttlingen, Germany
☎ + 49 7461 96490
Fax: + 49 7461 77921

LCN 460-3002-85/N

© 2002–2020 Symmetry Surgical Inc.

♻️ Revised 08/2020

ENGLISH

IMPORTANT INFORMATION

Please Read Before Use

Symmetry Surgical® OPTI-LENGTH® Spinal Instruments Symmetry Surgical® Spinal Instruments

Rx Only

Description

See outer label for a description of the package content.

These instruments are provided nonsterile and must be cleaned and sterilized before use, including initial use.

WARNING

Use of this instrument for purposes or in a manner other than those described herein will result in instrument damage or failure which could cause serious injury or death.

CAUTION: Improper care of hand held surgical instruments can result in unsatisfactory performance.

Inspection

Inspect each instrument before and after use to detect imperfections. Do not use damaged instruments; send them for repair or replacement.

Reprocessing Instructions

Refer to Symmetry Surgical Instructions for Use #204233 for updated information regarding reprocessing of the instrument.

Copies of this instruction sheet are available by requesting IFU #204233 from Symmetry Surgical Customer Service at:

Telephone: 800 251 3000

Fax: +1 615 964 5566

E-mail: customerservice@symmetrysurgical.com

Proper Care

All surgical instruments, particularly delicate and microsurgical instruments, must be handled with extreme care. Protect the working ends, sharps, and delicate tips at all times, particularly during transit, cleaning, lubrication, sterilization, and storage within the hospital. This helps insure that the instruments function properly when needed.

Surgical instruments will corrode and deteriorate if exposed to harsh chemicals and solutions. Do not expose them to acids, harsh detergents, or prolonged exposure to saline soaks.

To request a copy of our brochure, "The Care and Handling of Surgical Instruments", please contact your local Symmetry Surgical sales representative.

Service and Repair

For service and repair outside the United States, contact your local Symmetry Surgical representative.

Inside the United States, send instruments for service or repair to:

Symmetry Surgical Inc.
3034 Owen Drive,
Antioch, TN 37013 USA

Warranty

Symmetry Surgical warrants that this medical device is free from defects in both materials and workmanship. **Any other express or implied warranties, including warranties of merchantability or fitness, are hereby disclaimed. Suitability for use of this medical device for any particular surgical procedure should be determined by the user in conformance with the manufacturer's instructions for use. There are no warranties that extend beyond the description on the face hereof.**

OPTI-LENGTH® is a registered trademark of Symmetry Surgical

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Téléphone : 800 251 3000

Fax : +1 615 964 5566

E-mail : customerservice@symmetrysurgical.com

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