INSTRUCTIONS FOR USE: Scope CleanOR™ Scope and Trocar Cleaning Device EN

INDICATIONS FOR USE / CLINICAL BENEFITS

The Scope CleanOR is indicated to be used during endoscopic and laparoscopic surgery to keep the scope lens and trocar valves clean and free of debris.

CONTRAINDICATIONS

This device is not intended for use except as indicated.

CAUTIONS

- Verify compatibility. Inspect the endoscope / laparoscope and trocar for sharp edges that could interfere with the components of the Scope CleanOR (microfiber cloth and trocar swabs).
- Improper use of the Scope CleanOR may lead to patient injuries and / or damage of the endoscope / laparoscope and trocar.
- Trocar swabs should not be inserted on an angle or extended beyond the distal end of the trocar.

WARNINGS

- The Scope CleanOR was designed, tested and manufactured for single patient use only. After use, the Scope CleanOR should be disposed in accordance with local guidelines for medical waste.
- Device is sterile if the package is dry, unopened and undamaged. Do not use if the pouch is damaged or the packaged seal is broken.
- Do not re-sterilize the device. Re-sterilization may compromise the integrity of the device and create the risk of contamination and unintended patient infection.
- Discard the Scope CleanOR if mishandling has caused possible damage or contamination, or if the device has passed its expiry date.

INSTRUCTIONS FOR USE

- Using sterile technique, remove the Scope WipOR microfiber cloth and the two Trocar CleanOR swabs from the packaging. Inspect the device for the damage or contamination, and discard if any is noted. Otherwise, place it on sterile field.

During the procedure, anytime the scope is removed from the body:

- Clean the Scope lens using the Scope WipOR microfiber cloth.
- To remove debris from the trocar valves use the small trocar swab for 5-8mm trocars, and the larger trocar swab for 10-12mm trocars. NOTE: Trocar must be fully inserted into the abdominal cavity prior to use of the Trocar CleanOR.
- Select the appropriate trocar swab and insert its foam head into the trocar valve. Advance the swab into the valve, but do not extend the foam head beyond the distal end of the trocar. Move the swab back and forth to absorb any fluid and clean debris from the cannula. Do not release the trocar swab. NOTE: The shafts of the trocar swab are radiopaque. Trocar CleanOR must be used under direct vision.
- Check the cannula for free passage. If this is not the case, repeat the process.
- Before disposing the trocar swabs and microfiber cloth visually inspect the devices to ensure they are intact and all components have been retrieved.
- Upon completion of the procedure, the Scope CleanOR should be discarded in biohazardous waste as per standard procedure.

Any serious incident that has occurred in relation to this device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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| | | | |
|---|--|---|--|
|  | Product conforms with requirements of European Medical Device Regulation 2017/745. |  | Caution: US Federal Law restricts this device to sale by or on the order of a physician. |
|  | Caution, consult accompanying documents. |  | Single Sterile Barrier System. |
|  | Do not re-sterilize. |  | Do not use if package is damaged. |
|  | Keep away from sunlight. |  | Consult Instructions for Use. |
|  | Keep dry. |  | Not made with natural rubber latex. |
|  | Method of sterilization using ethylene oxide. |  | Do not reuse. |
|  | Sterile single barrier system with protective packaging outside. |  | Medical Device. |
|  | Use-by date. |  | Trademark protected. |
|  | Batch Code |  | Quantity |
|  | Manufacturer |  | Catalogue number |
|  | Unique device identifier |  | Date of manufacturer |
|  | Authorized representative in the European Community/European Union |  | Indicates the authorized representative in Switzerland |
|  | Product conforms with the requirements of UK MDR 2002 | | |

Distributed exclusively in USA by: Aspen Surgical Products,Inc. 6945 Southbelt Dr. SE, Caledonia, MI 49316, USA. aspensurgical.com. Phone: 616-698-7100, Toll-Free: 888-364-7004

 OBELIS SA, Bd Général Wahis 53, 1030 Brussels, BELGIUM, Tel: +(32) 2 7325954

 OBELIS SWISS GmbH, Ruessenstrasse 12, 6340 Baar/ZG, Switzerland. Tel: +41 41 544 15 26

UK Responsible Person: OBELIS UK LTD, Sandford Gate, East Point Business Park, Oxford, OX4 6LB, United Kingdom. Tel: +44.1491.378012

Issued date: 14-11-2025

PM_SC_101_V9