



Instructions For Use for *ClingTouch™* Sterile Disposable Monitor Cover

 aspen surgical®

INTENDED PURPOSE / INTENDED USE

This is a medical device used as sterile protective covering for imaging systems, cords, and keyboards to minimize cross contamination.

USE INSTRUCTIONS

1. Open pouch.
2. Remove first adhesive liner from open edge of monitor cover.
3. Pick cover up at corners, and allow cover to unroll.
4. Press the adhesive side of the monitor cover against the top of the monitor. Remove second adhesive liner from bottom edge of cover.
5. If necessary, gently smooth cover. Press the adhesive on the base of the cover against the base of the monitor to secure cover in place.



This product is single-use only.
Do not reuse. Do not resterilize. Reuse, reprocessing or re-sterilization of the single use devices may create a risk of cross contamination, patient infection, and / or malfunction of the device.

After use, dispose of single-use components as infectious waste.
Do not use if sterile package is damaged.

For translations of this instruction, go to aspensurgical.com/ifu.

STERILE EO



Do not use if sterile package is damaged.



Do not re-use.



Do not re-sterilize

EC REP

Emergo Europe
Prinsessegracht 20
2514 AP The Hague
The Netherlands



Aspen Surgical Products, Inc.
6945 Southbelt Dr. SE, Caledonia, MI 49316
phone +1 616.698.7100 toll-free +1 888.364.7004
aspensurgical.com

CE
2797

Made in Mexico
1-041-0180 Rev 3
14FEB2022