BOOKWALTER® Rectal Kit

Symmetry Surgical Inc.
3034 Owen Drive
Antioch, TN 37013 USA
☎ 1-800-251-3000
FAX: 1-615-964-5566
www.symmetrysurgical.com

EC REP

Symmetry Surgical GmbH
Maybachstraße 10
78532 Tuttlingen, Germany
☎ F r @ x r u i @ u r @ g
F r @ x r u i x x @ i

LCN 500-4546-85/L

© 2002–2015 Symmetry Surgical
♻ Revised 06/1t

ENGLISH

IMPORTANT INFORMATION

Please Read Before Use

BOOKWALTER® Rectal Kit

Description

All components of the BOOKWALTER® Rectal Kit, catalog no. 50-4546, are fabricated of stainless steel. The BOOKWALTER® Rectal Kit contains the following instruments:

Catalog No.	Description
50-4584	BOOKWALTER® Round Ring, Small, 22 cm inner diameter
50-4585	BOOKWALTER® Rectal Blades, Small, 22 mm x 7 cm (2 each)
50-4586	BOOKWALTER® Rectal Blades, Large, 23 mm x 10 cm (2 each)

The BOOKWALTER® Rectal Kit is designed to be used in conjunction with the following BOOKWALTER® Instruments which are available separately and as part of the BOOKWALTER® Retractor Kit, catalog no. 50-4545:

Catalog No.	Quantity Required	Description
50-4552	1 each	Horizontal Bar (use only for buie position, DO NOT use for lithotomy position—see “Horizontal Flex Bar” for lithotomy position)
50-4554	1 each	Post Coupling
50-4581	1 each	Table Post
50-4579	*	Ratchet Mechanism (2 each supplied with BOOKWALTER® Retractor Kit)
50-4580	*	Tilt Ratchet Mechanism (3 each supplied with BOOKWALTER® Retractor Kit)

* A total of four ratchet mechanisms (one for each rectal blade) is required. Use any combination of catalog nos. 50-4579 and 50-4580 to achieve a total of four ratchet mechanisms.

The following instrument is available only separately, and can be used when the patient is in either the buie position or the lithotomy position:

50-4582	1 each	Horizontal Flex Bar
---------	--------	---------------------

Indications

When used in conjunction with the appropriate BOOKWALTER® Components, the BOOKWALTER® Rectal Kit is designed for retraction in rectal surgery and small incision surgery.

WARNINGS

The retractor system can exert a great amount of force on the wound edges and on tissues. Release the retractor for short periods of time or shift the blades to relieve the pressure on any one point.

If the table is to be tilted toward the side holding the post, it is important to place adequate padding between the table post and the patient. Pressure necrosis or a nerve palsy could result if the patient is pressed against the unpadded bar for a long period.

Precautions

If using monopolar electro-surgical equipment while using the BOOKWALTER® Rectal Kit, consult the basic rules of usage concerning electro-surgery (see EN 60 601-2-2), as well as the manufacturer's instructions for use regarding specific warnings and conditions.

Be sure to electrically isolate the patient from (high frequency leakage currents to) the operating table, in accordance with the instructions provided by the manufacturer of the electro-surgical equipment.

Ensure that the patient does not come into contact with metal parts that are earthed or that have an appreciable capacitance to earth (for example: operating table, supports), in accordance with the instructions provided by the manufacturer of the electro-surgical equipment.

Avoid small-area contacts with objects having a low impedance to earth at high frequencies, as unwanted burns may result.

Instructions for Use

Please read the instruction manual supplied with the BOOKWALTER® Retractor or the product insert supplied with the table post, horizontal flex bar, post coupling, ratchet mechanisms, or tilt ratchet mechanisms before proceeding.

WARNING: The retractor system can exert a great amount of force on the wound edges and on tissues. Release the retractor for short periods of time or shift the blades to relieve the pressure on any one point.

Reprocessing Instructions

Refer to Symmetry Surgical® Instructions for Use #204233 for updated information regarding reprocessing the instrument. See CAUTIONS below.

Copies of this instruction sheet are available by requesting IFU #204233 from Symmetry Surgical Customer Service at:

Telephone: 800 251 3000

Fax: +1 615 964 5566

E-mail:

ssi-customerservice@symmetrysurgical.com

CAUTION: If flash autoclaved, take special care to cool off the vertical table post, horizontal flex bar, and post coupling with sterile, distilled water. These items will retain heat and require cooling before assembly.

CAUTION: To ensure proper function do not lubricate the Horizontal Flex Bar catalog no. 50-4582.

Service and Repair

For service or repair outside the United States, contact your local Symmetry Surgical representative.

Inside the United States, send instruments for service or repair to:

Symmetry Surgical
3034 Owen Drive,
Antioch, TN 37013 USA

Always include a repair purchase order number and a written description of the problem.

Warranty

Symmetry Surgical® warrants that this medical device is free from defects in both materials and workmanship. **Any other express or implied warranties, including warranties of merchantability or fitness, are hereby disclaimed. Suitability for use of this medical device for any particular surgical procedure should be determined by the user in conformance with the manufacturer's instructions for use. There are no warranties that extend beyond the description on the face hereof.**

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BOOKWALTER® is a trademark of