

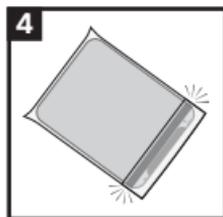
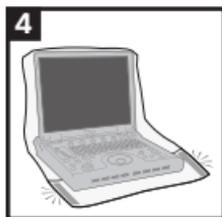
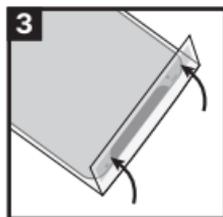
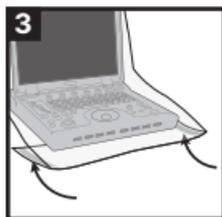
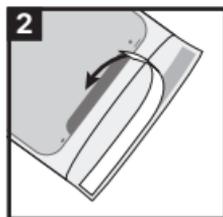
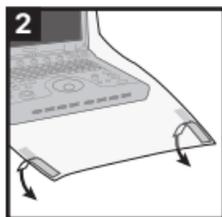
INSTRUCTIONS FOR USE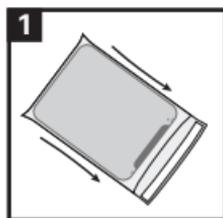

Sterile System Cover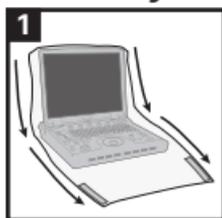

English

Intended Use

This is a medical device used as protective covering for imaging systems, cords, monitors, and keyboards to minimize cross contamination.

Protective covers help prevent transfer of microorganisms, body fluids, and particulate matter to the patient and healthcare provider during procedures using ultrasound transducers/imaging instruments and systems.

Intended User/Patient Target Groups

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

Instructions for Use

Prepare system per operational manual.

1. Place cover over system.
2. Peel off lining from tape.
3. Tuck excess drape.
4. Seal overlapping cover.
5. The device is intended for continuous use for less than 60 minutes.
6. Do not use if package is damaged.

Warning

Single Use Devices Only: Reusing or reprocessing single use devices may cause damage to the device which in turn may cause unnecessary harm to the user and/or patient. Reuse or reprocessing is not recommended.

Disposal

Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional, and/or local laws and regulations. Notice: After use or upon determining product can no longer be used due to wear or damage, safely discard in accordance with established procedures for biohazardous waste.

NOTICE: Any serious incident that has occurred in relation to the device should be reported to Aspen Surgical Products, Inc., the governing body, and the Competent authority of the member state in which the incident occurred.

Notice: For applicable IFU/additional information: Visit aspensurgical.com

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Aspen Surgical Products Inc.
Emergo Consulting (UK) Limited