Castaneda Infant Cardiovascular Kit

Symmetry Surgical Inc.
3034 Owen Drive,
Antioch, TN 37013 USA
☎ 1-800-251-3000
Fax: 1-615-964-5566
www.symmetrysurgical.com

EC REP

Symmetry Surgical GmbH
Maybachstraße 10
78532 Tuttlingen, Germany

☎ + 49 7461 96490
Fax: + 49 7461 77921

C € 2797

(These instructions pertain to both
Class I and Class II devices)

LCN 300-9025-85/P
© 2002–2020 Symmetry Surgical

♻️ Revised 08/2020

ENGLISH

IMPORTANT INFORMATION

Please Read Before Use

Castaneda Infant Cardiovascular Kit

Indications

The Castaneda Infant Cardiovascular Kit was specifically designed for open heart procedures on the newborn. It may also be used on small adult vessels.

Description

The Castaneda Infant Cardiovascular Kit, catalog no. 30-9025, includes the following instruments (see Figure 1):

Instruments	Catalog No.
A. Two Castaneda Vascular Forceps, 6 in.	30-8500
B. Castaneda Multi-Purpose Clamp, 30°, 5 in.	30-9000
C. Castaneda Multi-Purpose Clamp, 50°, 5 in.	30-9001
D. Castaneda Mixer Clamp, Smooth Jaw, 5¼ in.	19-8150
E. Castaneda Partial Occlusion Clamp, Spoon Type, 4¾ in.	30-9002
F. Castaneda Sternal Retractor, Large	50-8061
G. Castaneda Anastomosis Clamp, 6 in. Jaws 1.8 cm long, 7 mm deep	30-9004
H. Castaneda Anastomosis Clamp, 6 in. Jaws 2.2 cm long, 9 mm deep	30-9005
I. Castaneda Anastomosis Clamp, 6 in. Jaws 1.5 cm long, 5 mm deep	30-9003
J. Castaneda Sternal Retractor, Small	50-8060
K. Castaneda Instrument Case	21-8010

Cleaning

Disassemble and scrub the instruments and parts thoroughly using a soft brush and a mild detergent. Remove all traces of blood and debris. Make sure all moveable parts are cleaned thoroughly to prevent debris from interfering with movement. It is recommended that the instruments and parts be ultrasonically cleaned.

Lubrication

It is extremely important that moveable parts be properly lubricated to keep these parts functional. It is recommended that all components being steam sterilized be immersed in PRESERVE® Instrument Lubricant (catalog no. 43-4035).

Sterilization

The instruments are provided nonsterile and must be sterilized before use, including first use. The instruments can be steam or gas sterilized. Refer to the sterilizer manufacturer's instructions for correct time, temperature and pressure settings.

Service and Repair

For service and repair outside the United States, contact your local Symmetry Surgical representative.

Inside the United States, send instruments for service or repair to:

Symmetry Surgical Inc.
3034 Owen Drive,
Antioch, TN 37013 USA

Warranty

Symmetry Surgical warrants that this medical device is free from defects in both materials and workmanship. **Any other express or implied warranties, including warranties of merchantability or fitness, are hereby disclaimed. Suitability for use of this medical device for any particular surgical procedure should be determined by the user in conformance with the manufacturer's instructions for use. There are no warranties that extend beyond the description on the face hereof.**

PRESERVE® is a registered trademark of Symmetry Surgical

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© 2002–2020 Symmetry Surgical Inc.