Beach Chair Facial Positioning

Non-Sterile

Bethesda Beach Chair Face Mask 711000

Erin Beach Chair Face Mask 711001

Indy Beach Chair Face Mask 711003

Universal Head Restraint 711004

Beach Chair Head and Chin Strap 711005

Aspen Surgical Products, Inc.
6945 Southbelt Drive SE
Caledonia, MI 49316 USA
Phone +1 616-698-7100
Toll-Free 888-364-7004
aspensurgical.com

Emergo Europe
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands

For Country of Origin See Product Label

English

Intended Use

Intended to support and maintain the position of an anesthetized patient during surgery.

Intended User/ Patient Target Groups

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

Instruction for Use:

- Place mask over patient's face.
- Secure straps or buckles into receiving clips on positioning device.
- Tighten straps if necessary (see precaution).
- Device is intended for limited contact duration, less than or equal to 8 hours, less if damaged.
- Discard after use.

Precautions:

Compatibility Chart:	
For Use With:	Aspen Surgical Products
Smith & Nephew® T-MAX™ Beach Chair 7210996	Erin Beach Chair Face Mask 711001, Non-Sterile
ALLEN® FLEXIFORM™ Head Rest A-90017	Indy Beach Chair Face Mask 711003
SchureMed™ Beach Chair 800-0004, Stryker® Shoulder Chair 0Z7519475 and 0Z 7523075, and Skytron® Beach Chair	Bethesda Beach Chair Face Mask 711000, Non-Sterile
SKYTRON® Beach Chair, SCHUREMED™ Beach Chair, STERIS® Beach Chair, STRYKER® Shoulder Chair 0Z 7519475 and 0Z 7523075	Beach Chair Head and Chin Strap 711005
Allen® Beach Chair Positioner	Universal Head Restraint 711004

- Ensure mask is properly aligned over eyes, nose and chin.
- Ensure that skin & hair are away from straps when securing device.
- Ensure that the straps or buckles are securely fastened to the positioning device.
- Do not overtighten straps on mask.
- This device is single use only.
- Do not sterilize.

Warning

Reuse of device could result in infection/contamination and/or device failure which could lead to patient harm.

Disposal

Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional and/or local laws and regulations.

Notice to Users and/or Patients in EU

Any serious incident that has occurred in relation to the device should be reported to Aspen Surgical Inc., the governing body, and the Competent authority of the member state in which the incident occurred.

Smith & Nephew T-MAX is a registered trademark of Smith & Nephew PLC (Smith & Nephew). Aspen Surgical Products, Inc. is not affiliated with, endorsed by, or sponsored by Smith & Nephew.

Allen® and Flexiform™ are registered trademark or trademarks of Allen Medical Systems, Inc. ("Allen Medical"). Aspen Surgical Products, Inc. is not affiliated with, endorsed by, or sponsored by Allen Medical.

Skytron® is a registered trademark of Skytron, LLC. ("Skytron"). SchureMed™ is a trademark of Schuerch Corporation ("Schuerch"). Steris® is a registered trademark of Steris Inc. ("Steris"). Stryker® is a registered trademark of Stryker Corporation ("Stryker"). Aspen Surgical Products, Inc. is not affiliated with, endorsed by, or sponsored by Skytron, Schuerch, Steris, or Stryker. SchureMed is a trademark of Schuerch Corporation ("Schuerch"). Skytron is registered trademark of Skytron, LLC. ("Skytron"). Stryker is a trademark of Stryker Corporation ("Stryker"). Aspen Surgical Products, Inc. is not affiliated with, endorsed by, or sponsored by Schuerch, Skytron, or Stryker.

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Ergemo Consulting (UK) Limited