BOOKWALTER® Retractor Kit II

Instruction Manual

Symmetry Surgical Inc.
3034 Owen Drive,
Antioch, TN 37013 USA
☎ 1-800-251-3000
Fax: 1-615-964-5566
www.symmetrysurgical.com

EC REP

Symmetry Surgical GmbH
Maybachstraße 10
78532 Tuttlingen, Germany
☎ +49 7461 96490
Fax: +49 7461 77921

LCN 169587-001/P
© 2002–2015 Symmetry Surgical
♻️ Revised 10/15

ENGLISH

IMPORTANT INFORMATION

Please Read Before Use

BOOKWALTER® Retractor Kit II Instruction Manual

Description

The BOOKWALTER® Retractor Kit II (catalog no. 50-4542) is a self-retaining retraction system. The retractor, which attaches directly to the operating room table rail over paper or linen drapes, provides multidirectional exposure. The instruments are supplied nonsterile and must be sterilized before use, including initial use. The kit includes (see Figure 1):

- A.** Table Post 1 in. (2.5 cm) (1 each)
50-4581
- B.** Horizontal Bar (1 each)
50-4552
- C.** Horizontal Flex Bar (1 each)
50-4582
- D.** Post Coupling
1 in. x 1 in. (2.5 cm x 2.5 cm) (1 each)
50-4554
- E.** Medium Oval Ring
10½ in. I.D. x 14½ in. I.D. (26.7 cm x 36.8 cm)
(1 each)
50-4557
- F.** Segmented Ring (1 each)
50-4583
- G.** Malleable Retractor Blades
2 in. x 6 in. (5.1 cm x 15.2 cm) (2 each)
50-4561
- H.** Malleable Retractor Blade
1½ in. x 6 in. (3.8 cm x 15.2 cm) (1 each)
50-4562
- I.** Malleable Retractor Blade
3 in. x 6 in. (7.6 cm x 15.2 cm) (1 each)
50-4567
- J.** Kelly Retractor Blades
2 in. x 3 in. (5.1 cm x 7.6 cm) (2 each)
50-4568
- K.** Kelly Retractor Blade
2 in. x 4 in. (5.1 cm x 10.2 cm) (1 each)
50-4569

- L.** Kelly Retractor Blade
2 in. x 5 in. (5.1 cm x 12.7 cm) (1 each)
50-4570
- M.** Kelly Retractor Blade
2 in. x 6 in. (5.1 cm x 15.2 cm) (1 each)
50-4571
- N.** Balfour Retractor Blade (1 each)
50-4576
- O.** Harrington Retractor Blade (1 each)
50-4566
- P.** Gelpi Retractor Blade (1 each)
50-4577
- Q.** Tilt Ratchet Mechanisms (6 each)
50-4580
- R.** Ratchet Mechanisms (2 each)
50-4579

In addition, the following BOOKWALTER® Components may be ordered separately:

- Table Post ¾ in. (2.2 cm)
50-4551
- Table Post 1 in. (2.5 cm)
50-4581
- Vertical Extension Bar 1 in. x 12 in.
(2.5 cm x 30.5 cm)
50-4553
- Horizontal Flex Bar
50-4582
- Horizontal Bar
50-4552
- Post Coupling ¾ in. x 1 in. (2.2 cm x 2.5 cm)
50-4555
- Post Coupling 1 in. x 1 in. (2.5 cm x 2.5 cm)
50-4554
- Large Oval Ring 12 in. I.D. x 18½ in. I.D.
(30.5 cm x 47.0 cm)
50-4556
- Medium Oval Ring 10½ in. I.D. x 14½ in. I.D.
(26.7 cm x 36.8 cm)
50-4557
- Small Oval Ring 8 in. I.D. x 12¼ in. I.D.
(20.3 cm x 31.1 cm)
50-4558
- Medium Round Ring 10½ in. I.D. (26.7 cm)
50-4559
- Large Round Ring 12½ in. I.D. (31.8 cm)
50-4560
- Segmented Ring
50-4583

Malleable Retractor Blade 1½ in. x 6 in.
(3.8 cm x 15.2 cm)
50-4562

Malleable Retractor Blade 2 in. x 6 in.
(5.1 cm x 15.2 cm)
50-4561

Malleable Retractor Blade 3 in. x 6 in.
(7.6 cm x 15.2 cm)
50-4567

Kelly Retractor Blade 2 in. x 1½ in.
(5.1 cm x 3.8 cm)
50-4565

Kelly Retractor Blade 2 in. x 2½ in.
(5.1 cm x 6.4 cm)
50-4572

Kelly Retractor Blade 2 in. x 3 in.
(5.1 cm x 7.6 cm)
50-4568

Kelly Retractor Blade 2 in. x 4 in.
(5.1 cm x 10.2 cm)
50-4569

Kelly Retractor Blade 2 in. x 5 in.
(5.1 cm x 12.7 cm)
50-4570

Kelly Retractor Blade 2 in. x 6 in.
(5.1 cm x 15.2 cm)
50-4571

Harrington Retractor Blade
50-4566

Balfour Retractor Blade
50-4576

Gelpi Retractor Blade
50-4577

Tilt Ratchet Mechanisms
50-4580

Ratchet Mechanisms
50-4579

Basic Kit Storage Case
50-4544

Inspection

Inspect each instrument before and after use to detect imperfections. Do not use damaged instruments; send them for repair or replacement.

WARNINGS

The retractor system can exert a great amount of force on the wound edges and on tissues. Release the retractor for short periods of time or shift the blades to relieve the pressure on any one point.

If the table is to be tilted toward the side holding the post, it is important to place adequate padding between the table post and the patient. Pressure necrosis or a nerve palsy could result if the patient is pressed against the unpadded bar for a long period.

Precautions

As with any self-retaining retractor, caution should be taken not to exert undue retraction pressure on the iliopsoas muscle to prevent possible femoral nerve palsy.

If using monopolar electrosurgical equipment while using the BOOKWALTER® Retractor Kit II, consult the basic rules of usage concerning electrosurgery (see EN 60 601-2-2), as well as the manufacturer's instructions for use regarding specific warnings and conditions.

Be sure to electrically isolate the patient from (high frequency leakage currents to) the operating table, in accordance with the instructions provided by the manufacturer of the electrosurgical equipment.

Ensure that the patient does not come into contact with metal parts that are earthed or that have an appreciable capacitance to earth (for example: operating table, supports), in accordance with the instructions provided by the manufacturer of the electrosurgical equipment.

Avoid small-area contacts with objects having a low impedance to earth at high frequencies, as unwanted burns may result.

Instructions for Use

Attaching the Vertical Table Post

NOTE: Be sure the operating table side rail does not have excess material draped over it. The clamp of the vertical table post is designed to hold no more than two double thicknesses of cotton drapes.

Pull out the recessed lever located at the top of the vertical post (Figures 2 and 3) and fully open the rail clamp by turning the lever counterclockwise. Attach the clamp over the drapes as shown in Figure 4. Retighten the clamp to secure the post by turning the lever clockwise (Figure 5).

WARNING: If the table is to be tilted toward the side holding the post, it is important to place adequate padding between the table post and the patient. Pressure necrosis or a nerve palsy could result if the patient is pressed against the unpadded bar for a long period.

Attaching the Post Coupling, Horizontal Bar, and Ring

NOTE: For convenience and correct orientation of the ring, it is recommended that this procedure be performed after the skin incision has been made. However, this procedure may be performed at any time.

Slide the horizontal bar into either of the holes in the post coupling. Slide the post coupling and horizontal bar assembly over the top of the vertical post, to its approximate location, and lightly tighten the coupling. The position of the post coupling lever on the vertical post should be convenient to the surgeon. Make any final positioning adjustments before tightening the post coupling (Figure 6).

Center the ring over the operative site and adjust the post coupling to obtain the desired position. Hold the ring in position while locking in place, as shown in Figure 7. Make any final positioning adjustments with the ring attached over the wound.

If it is still necessary to reposition the post coupling, make sure the serrated teeth are disengaged fully before rotating the side with the horizontal bar.

CAUTION: The post coupling bolts never should be tightened unless the post coupling is attached to the bars. The slight deformity of the circular opening that might result could affect the smooth fit between the post coupling and the bars.

With the segmented ring, to remove or adjust a ring segment, turn the locking screw counterclockwise until the serrated teeth of the fittings disengage. Adjust the segment as required and retighten the locking screw.

CAUTION: Before fully tightening locking screw, be certain that the serrated teeth of the ring segment fittings mesh. Failure to do so may damage the fittings.

To remove the screw from a ring segment for cleaning, pull the screw through the fitting until it engages the threads. Unscrew and remove by turning counterclockwise.

Adjusting the Angle of the Ring

To adjust the angle of the ring when using the horizontal flex bar, release the universal joint lever on the horizontal flex bar (Figure 8) by turning the lever counterclockwise. Relaxing the universal joint assembly results in a variety of positioning options. Retighten the lever by turning clockwise.

Attaching the Retractor Blades

1. Place a ratchet mechanism onto a retractor blade.

NOTE: The ratchet mechanisms are designed to fit on any retractor blade.

2. Connect the ratchet mechanism onto the ring, as shown in Figure 9. To adjust the blade, press in the ratchet release and then push or pull the retractor to the desired setting.

Using the Tilt Ratchet Mechanism

To use the lifting capability of the tilt ratchet mechanism, simply depress the thumb lever to adjust the elevation for increased exposure (Figure 10).

Reprocessing Instructions

Refer to Symmetry Surgical Instructions for Use #204233 for updated information regarding reprocessing of the instrument. See CAUTIONS below.

Copies of this instruction sheet are available by requesting IFU #204233 from Symmetry Surgical Customer Service at:

Telephone: 800 251 3000

Fax: +1 615 964 5566

E-mail:

customerservice@symmetrysurgical.com

CAUTION: If the instruments are flash autoclaved, take special care to cool off the vertical table post, horizontal flex bar and post coupling with sterile, distilled water. These items will retain heat and require cooling before assembly.

CAUTION: To ensure proper function do not lubricate the Horizontal Flex Bar catalog no. 50-4582.

Service and Repair

For service or repair outside the United States, contact your local Symmetry Surgical representative.

Inside the United States, send instruments for service or repair to:

Symmetry Surgical Inc.
3034 Owen Drive,
Antioch, TN 37013 USA

Always include a repair purchase order number and a written description of the problem.

Warranty

Symmetry Surgical warrants that this medical device is free from defects in both materials and workmanship. **Any other express or implied warranties, including warranties of merchantability or fitness, are hereby disclaimed. Suitability for use of this medical device for any particular surgical procedure should be determined by the user in conformance with the manufacturer's instructions for use. There are no warranties that extend beyond the description on the face hereof.**

BOOKWALTER® is a registered trademark of Symmetry Surgical

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