EN ELECTROSURGICAL PENCIL; REUSABLE

- REF ESPR-EL

- Store At Room Temperature
- Colet fits electrodes with a shaft of .062” Min - .064” Max
- Device is rated for 4.0k Vpeak

WARNING: Do not use in the presence of flammable materials. Fire could result.

WARNING: Improper electrode installation may result in injury to the patient or operating room personnel by arcing at the electrode pencil connection.

WARNING: To avoid burn or shock injury, turn off the desiccator at the mains power.

WARNING: To avoid burn or shock injury, do not activate pencil while wiping electrode tip during use.

WARNING: Keep active accessories away from the patient when not in use. Accessories and electrodes should be placed in a clean, dry, nonconductive and highly visible area when not in use.

WARNING: Inspect prior to use. The device should be replaced if any physical damage such as nicks, cracks or cuts is noted on the handpiece or cable.

CAUTION: This product is NON-STERILE. The use of a sterile sheath or sterilizing prior to use is recommended.

CAUTION: Use the lowest possible setting to achieve the desired effect.

DIRECTIONS FOR USE:

- Place electrode into the handpiece. Ensure that the electrode is fully inserted.
- Connect the handpiece to the monopolar output receptacle of the electrourgical generator.
- To activate, depress the appropriate button for the desired effect: Yellow for Cut and Blue for Coag. Make sure the generator responds accordingly.
- After use, remove electrode and resterilize the pencil following the recommended procedure as follows.

CLEANING INSTRUCTIONS:

- Remove all gross matter (blood, tissue) by wiping down the pencil using a gauze pad or soft cloth dampened in warm tap water until all visible debris is removed.
- Wipe the pencil using a gauze pad or soft cloth dampened in an enzymatic detergent mixture diluted according to the manufacturer’s instructions. The pencil may also be gently scrubbed with a soft cleaning brush.
- IMPORTANT: Abrasive material should not be used in the area of the barcode when cleaning the device.
- Remove cleaning agents from pencil by wiping with a tap water dampened cloth.
- Dry pencil thoroughly.

STERILIZATION INSTRUCTIONS:

Number of uses when sterilized by steam autoclave: 25

- Steam autoclave at 250° F (121° C; gravity) for 20 minutes using an autoclavable pouch or wrapped cloth method. Consult the AAMI recommended practice for steam sterilization using the wrapped cloth method. Allow product to cool at least 30 minutes before use.
- Flash autoclave method is not recommended.

Medical Purpose/ Indication
Device is intended to be used in general electro surgical applications for cutting and coagulation during surgical procedures.

Operating Principle

The device is used in conjunction with an electro surgical generator and replaceable electrodes for the delivery of radiofrequency (RF) current. The device is to be used with a variety of monopolar electrodes for the coagulation and desiccation of tissue.

Patient Population

No restrictions

Site of Use and Site Conditions

Site of use – Soft tissue

Site conditions – Aseptic

Intended User Profile

This device is intended to be used by trained physicians or individuals familiar with electro surgery.

Intended Conditions for Use

- Education
- Trained physician, physicians assistant, nurse or nurse practitioner
 - Understands electro surgery and electro surgical techniques
 - Has read and understands supplied Instructions for Use or Device Labeling
 - Experience
 - Some training on techniques or training under surveillance/supervision

Permissible impairments

Mild reading vision impairment or corrected vision to 20/20

Intended conditions for use

- Doctor office, Surgery center or Hospital, intended for professional use only
- Lit, clear and unobstructed view at point of use
- Ambient Temperature 10° C to 40° C

Frequency of use

- The ESPR-EL is reusable and designed to withstand a maximum of 25 gravity autoclave sterilization cycles

Disposal

- Biohazard – Dispose of Pencil according to facility procedures

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