INSTRUCTIONS FOR USE: HolstOR™ Surgical Instrument Holder

EN

INDICATIONS FOR USE / CLINICAL BENEFITS

HolstOR is a surgical instrument holder, designed specifically to improve safety by reducing unnecessary instrument exchanges between surgeon and scrub personnel in the operation room.

CONTRAINDICATIONS

This device is not intended for use except as indicated.

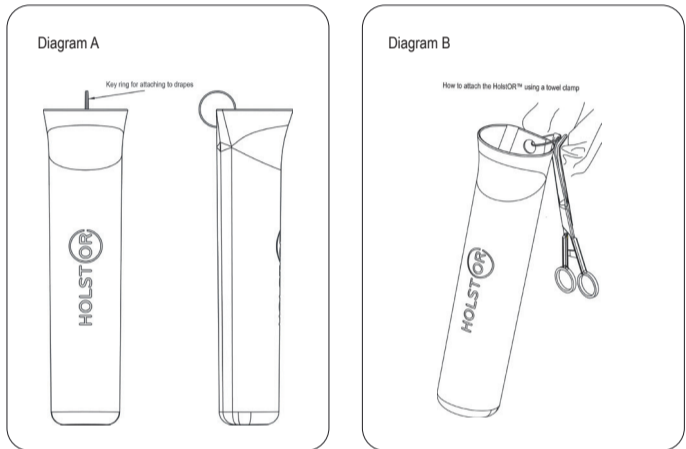
CAUTIONS / WARNINGS

- Choose the HolstOR size in relation to the surgical instruments used. Do not overload the instrument holder.
- Each HolstOR requires its own towel clamp, even when connected to another HolstOR.
- The device is intended for single use only. The design of this device may not perform as intended by the manufacturer if it is reused. The manufacturer cannot guarantee the performance, safety and reliability of a reprocessed device.

INSTRUCTIONS FOR USE

- Remove the HolstOR from the sterile packaging and place it on the sterile instrument table.
- Drape patient as required.
- Secure the HolstOR to the sterile patient drape using a towel, Kelly or artery clamp and supplied 4mm key ring. See diagram B.
- Ensure the HolstOR is adequately secured to the sterile patient drapes prior to inserting any instrumentation.
- The HolstOR will comfortably accommodate up to three (3) separate surgical instruments.
- Additional HolstOR instrument holders can be connected utilising the accessory clips located on the side of the HolstOR.
- After the procedure, discard the device in the approved medical waste container in accordance with institutional policy.

Any serious incident that has occurred in relation to this device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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

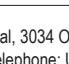
	Product conforms with the requirements of European Medical Device Regulation 2017/745.		Caution: US Federal Law restricts this device to sale by or on the order of a physician.
	Caution, consult accompanying documents.		Single Sterile Barrier System.
	Do not re-sterilize.		Do not use if package is damaged.
	Keep away from sunlight.		Consult Instructions for Use.
	Keep dry.		Not made with natural rubber latex.
	Method of sterilization using ethylene oxide.		Do not reuse.
	Single sterile barrier with protective packaging outside.		Medical Device.
	Use-by date.		Trademark protected.

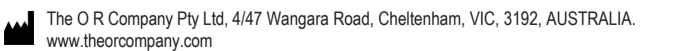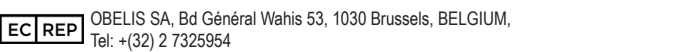Distributed exclusively in USA by: Symmetry Surgical, 3034 Owen Drive, Antioch, TN 37013, USA
Email: customerservice@symmetrysurgical.com, Telephone: USA & CANADA +1 800-251-3000 (Monday-Friday, 7:00AM - 5:30PM CST)

Issued date: 28-02-2022	PM_H_101_V4
-------------------------	-------------