

March 22, 2022

Dear Valued Customer:

Regarding the sterilization of Bookwalter components:

- The Post Coupling (50-4554) does not need to be disassembled for sterilization. Sterilize the Post Coupling in the open, untightened position.
- Locking knobs may remain in the segments of Segmented Rings (i.e. 50-4583) during sterilization if they are loosened such that all segments are easily movable with little effort.

Thank you for using the genuine Bookwalter® retractor. Please let me know if you have any questions.



Brandon Preske
Vice President Product Management
Symmetry Surgical, Inc.