For product information, contact:
Pour obtenir des informations sur le produit, veuillez
contacter :
Para obtener información del producto, póngase en
contacto con:

3230 Commerce Center Place
Louisville, Kentucky, 40211
(502) 813-5022 <> (800) 297-6344

See our products at

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ENGLISH

Description:

Olsen Medical® Single Use electrosurgical devices are sold as sterile, single use devices, unless otherwise noted. These devices are designed to function as active electrodes or as accessories to active electrodes in electrosurgical systems. This family of devices includes, but is not limited to: electrosurgical pens/pencils, electrode handles, monopolar and bipolar forceps, electrodes, cords and adapters, holsters, tip cleaning pads, or various combination packages of these devices.

Intended Use (per EU definition) /

Indications (per FDA definition):

Monopolar Cord and Adapters - Olsen Medical® electrosurgical cords and adapters are designed to bring high frequency electrical power from the electrosurgical generator to the active electrode and are intended for use with a maximum voltage of 3500 Volts. Bipolar Forceps and Cords- Olsen Medical® Bipolar Forceps and Cords are intended for use in coagulating tissue during surgical procedures and are intended for use with a maximum voltage of 500 Volts. Monopolar Forceps - Olsen Medical® Monopolar Forceps are intended for use in coagulating and cutting tissue during surgical procedures and are intended for use with a maximum voltage of 2000 Volts. Monopolar Electrodes - Olsen Medical® Electrodes are intended for use in coagulating and cutting tissue during surgical procedures and are intended for use with a maximum voltage of 5000 Volts. Pens/Pencils and Handles- Olsen Medical® pens/pencils and handles are designed to hold active electrodes and are intended for use with a maximum voltage of 3000 Volts. The distal end of the pencil has a 3/32" (2.36mm) collet that will accept a variety of electrosurgical electrodes with 3/32" (2.36mm) diameter shafts Non-Active Accessories- such as Holsters, Surgiwipes, Tip Cleaners, etc., are dependent non-active accessories intended to aid in the effective use of electrosurgical devices.

Handling:

1. Inspect package for damage before use. Discard if damaged or seal is broken.
2. Inspect device after opening in a sterile field. Discard if damaged.
3. Remove and discard the tip protector on all forceps before use.

4. Device use and application to be determined by physician.

Storage:

These devices should be stored in their original labeled boxes or in protected storage to prevent damage during handling. Take precautions to ensure that the sterile devices are stored in areas where it is completely dry, and protected from extremes of temperature and humidity.

Sterility:

Sterility is guaranteed up to the expiration date unless package is opened or seal is broken.

Sterilization:

Single Use Only (intended to be used on one patient during one procedure). Dispose after use. Do not resterilize. Reusing or reprocessing single use devices may cause damage to the device which in turn may cause unnecessary harm to the user and/or patient. Reuse or reprocessing is not recommended.

Notices:

- Electrosurgical equipment and accessories are intended to be used by health professional educated in their use.
- After Use or upon determining product can no longer be used due to wear or damage, safely discard in accordance with established procedures for biohazardous waste.
- Any serious incident that has occurred in relation to the device should be reported to Symmetry Surgical Inc., the governing body, and the Competent authority of the member state in which the incident occurred.
- Please refer to the manufacturer of the generator for warnings, precautions, contraindications, undesirable side-effect, measures to be taken, and limitations of use for the electrosurgical system and accessories
- For applicable IFU/additional information: Visit www.symmetrysurgical.com.

Caution:

- Inspect every device before use. Damaged electrosurgical devices may cause electrical burns to the patient or physician.
- Active electrosurgical devices must be placed away from the patient when not in use. A holster is recommended.
- Use a wet sponge to remove eschar build up from the active electrode tips. A tip cleaner can be used with monopolar devices.
- Consult RF generator Owner's Manual. Use only in the active RF generator output connections.
- Do not perform electrosurgery in the presence of flammable anesthetics or other flammable gases, fluids or objects, or in the presence of oxidizing agents, as fire could result.
- Avoid unnecessary or prolonged activation. Activate only when touching or immediately next to the intended point of contact.
- Non-insulating switching contacts of hand controlled forceps are not to touch or be in close proximity to the skin or bodily tissue to prevent patient/surgeon burn.
- Electrodes must be compatible with the pen/pencil or handle and ensure that the insulated sleeve fits securely inside the handle. Improperly seated electrode may cause burn or injury.
- Do not bend, cut, or modify electrode before or during surgery.

- Keep the voltage/power settings at the lowest possible level to achieve the desired effect.
- For Manually Extendable Electrocautery Apparatus, do not attempt to dislodge or remove the fixed extendable electrode to avoid breaking the device.
- After use, safely discard in accordance with established procedures for biohazardous waste.

Warning:

Improper use of any element in electrosurgical systems may cause serious burn, shock, and fire or explosion hazard. The use of the device with safely functioning and compatible handles, cords, electrosurgical generators and accessories is the user's responsibility. Read and understand all precautions and instructions before attempting to use any electrosurgical devices. This product is for SINGLE USE ONLY; DO NOT RESTERILIZE. Use aseptic technique in all phases of handling. Olsen Medical Single Use devices have not been designed to undergo or withstand any form of alteration, such as disassembly, cleaning or resterilization, after a single patient use. Reuse can potentially compromise device performance and any usage beyond the design intent of this single use device may result in unpredictable loss of functionality.

Warning:

Avoid HF output mode/power settings where maximum output voltage may exceed the stated maximum voltage.

Warning:

Do not make any modification to these devices. Modification may cause burn or injury.

Warning:

Due to concerns about the carcinogenic and infectious potential of electrosurgical byproducts (such as tissue smoke plume and aerosols which may include toxic gasses and vapors, live and dead cellular material, and viruses), protective eyewear, filtration masks, and effective smoke evacuation equipment should be used in both open and laparoscopic procedures. Contact Symmetry Surgical at customerservice@symmetrysurgical.com for additional information or to inquire about our smoke evacuation solutions.

Warranty:

All products are warranted to be free from defects in workmanship and materials when used for its intended purpose. Any product that proves defective in workmanship or materials will be repaired or replaced at our discretion free of charge. Repair, modification, or alteration to any product that is performed by anyone not authorized by Olsen Medical® will result in the immediate loss of warranty.

OLSEN® is a registered trademark of Olsen Medical.

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