# Symmetry Surgical® Tissue Approximator

**Symmetry Surgical Inc.**  
3034 Owen Drive  
Antioch, TN 37013 USA  
☎ 1-800-251-3000  
Fax: 1-615-964-5566  
[www.symmetrysurgical.com](http://www.symmetrysurgical.com)

 **symmetry surgical**  
Symmetry Surgical GmbH  
Maybachstraße 10  
78532 Tuttlingen, Germany  
☎ +49 7461 96490  
Fax: +49 7461 77921

**IFU-DG520 Rev A**  
© 2018/07 Symmetry Surgical

**ENGLISH****Symmetry® Tissue Approximator****INSTRUCTIONS FOR USE****Product Description**

The Symmetry® Tissue Approximator approximates and permits grasping of two tissue areas together to allow for stapling or suturing. The Symmetry® Tissue Approximator is an instrument consisting of three jaws. The outer two jaws enable the user to secure tissue to the central jaw. The jaws operate independently through a trigger mechanism, which allows the user to manipulate and approximate the tissue during a surgical procedure. The visually coded jaws are connected to the same color-coded control triggers located on the handle. The triggers may also be discerned using solely tactile feedback. Compression of the trigger opens the corresponding jaw. Both triggers may be operated individually or simultaneously.

**ATTENTION**

- Read and understand all information contained in these Instructions for Use before using the Symmetry® Tissue Approximator.
- The Symmetry® Tissue Approximator must be cleaned and sterilized prior to each procedure.
- Care must be taken to avoid dropping or mishandling this instrument.

### **Indications for Use**

The Symmetry® Tissue Approximator is intended to grasp and hold the tissues for approximation and in preparation for suturing or stapling in a variety of laparoscopic / endoscopic or open surgical procedures.

### **Contraindications**

Do not use the Symmetry® Tissue Approximator where hemostasis cannot be verified visually after application.

Laparoscopic or open use of the Symmetry® Tissue Approximator is contraindicated whenever surgical techniques or use of a grasping device are contraindicated for any reason.

### **Warning**

- A thorough understanding of the principles and techniques involved in laser and electro-surgical procedures is essential to avoid shock and burn hazards to patients, operators, and to the instruments.
- Avoid any harsh materials that can scratch or mar the instrument surface.
- Prior to use, clean and sterilize all reusable components per the instructions.
- As with any foreign material, the presence of bacterial contamination may enhance bacterial infectivity.

### **Precautions**

- Endoscopic/laparoscopic surgical procedures should be performed only by persons having adequate training and familiarity with laparoscopic techniques. Consult the medical literature relative to techniques, complications, and hazards prior to performing any laparoscopic procedures.

### **INSTRUMENT CLEANING AND STERILIZATION**

Recommendation: Gloves should be worn when cleaning and handling contaminated instruments. All institutional guidelines for infection control should be followed.

**Be sure to remove the flush port cap during cleaning and sterilization and to place back on before use. The flush port cap must be steam sterilized with the instrument.**

**Follow instructions in accordance to the General Reprocessing Instructions LCN-204233.**

## Disassembly instructions

**Figure 1.**

**Turn collar counterclockwise to loosen and remove handle**

**Figure 2.**

**Slide handle completely off, leave off for cleaning**

### Assembly instructions

**Figure 3.**  
Slide handle back into position

**Figure 4.**  
Retighten the collar to securely lock the handle.

## INSTRUMENT INSPECTION AND WARRANTY

This instrument should be inspected and tested after each autoclave cycle and prior to each use. If the instrument fails to function as intended, contact Symmetry Surgical Customer Service immediately for repair/replacement. Do not attempt to repair or modify this device.

Instrument warranty is void if service / repair is not performed by an authorized Symmetry Surgical Service Representative.

**DISPOSAL/SERVICE** Adhere to national regulations when disposing of or recycling the product, its components and its packaging. For service outside the United States, contact your local Symmetry Surgical representative. Inside the United States, call Symmetry Surgical Customer Service: 1-800-251-3000.

## WARRANTY

Symmetry Surgical warrants that this medical device is free from defects in both materials and workmanship for one (1) year from the date of purchase. **Any other express or implied warranties, including warranties of merchantability or fitness, are hereby disclaimed. Suitability for use of this medical device for any surgical procedure should be determined by the user in conformance with the manufacturer's instructions for use. There are no warranties that extend beyond the description on the face hereof. Abuse or misuse of the product or failure to comply with the instructions for use shall void this warranty.**

Copies of the IFU are available by requesting them from Symmetry Surgical Customer Service at: Phone: 1-800-251-3000

**Email:** [customerservice@symmetrysurgical.com](mailto:customerservice@symmetrysurgical.com)

**Website:** [www.symmetrysurgical.com](http://www.symmetrysurgical.com)

## CAUTION

USA federal law restricts the use of this device to sale by, or on the order of a physician.

Symbol Definitions:

Symbol for Manufacturer

Date of Manufacture


Lot number or batch number

Catalog number

Quantity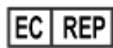

Authorized representative in the European Community

Conformance to 93/42/EEC

Non-Sterile

Caution, Consult Instructions for Use

Federal law restricts this device to sale by or on the order of a physician

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