OPTI-LENGTH®

MICROSECT® Shaver

MICROSECT® Shaver

Symmetry Surgical
3034 Owen Drive
Antioch, TN 37013 USA
☎ 1-800-251-3000
Fax: 1-615-964-5566
www.symmetrysurgical.com

[EC] [REP]

Symmetry Medical Sheffield
Beulah Road
Sheffield, England
S6 2AN
☎ +44-114-285-5881
Fax: +44-114-233-6978

LCN 183031-001/L
© 2002–2012 Symmetry Surgical
 Revised 07/13

ENGLISH

IMPORTANT INFORMATION

Please Read Before Use

OPTI-LENGTH® MICROSECT® Shaver MICROSECT® Shaver

Rx Only

Indications

The OPTI-LENGTH MICROSECT and MICROSECT Shavers are intended for microsurgical subarticular, sublaminar, or foraminal decompression in the spine.

Description

These instruments are provided nonsterile and must be cleaned and sterilized before use, including initial use.

OPTI-LENGTH MICROSECT and MICROSECT Shavers are available in varying sizes and are available separately. Refer to outer label for a description of the package content.

Information for Use

The thin footplate allows access into compromised spaces while reducing the risk of compression of underlying structures. The footplate is tapered to allow progressive bone removal with each shaving action, and to maximize efficiency as the microsurgical instrument is advanced into the area to be decompressed. The reversed cutting motion facilitates removal of bone away from the neurological structures.

WARNINGS

Use of this microsurgical instrument for purposes or in a manner other than those described will cause instrument damage or failure that could result in serious patient injury or death.

CAUTIONS

The tip of this microsurgical instrument can bend or fracture when overstressed or misused.

Minimize cortical bone thickness with alternate surgical techniques, such as drilling, to reduce the force necessary to cut tissue. Improper use of this microsurgical instrument on thick spinal or cranial bone can cause damage or breakage.

Do not twist or pry this microsurgical instrument while cutting as this can overstress the tip, leading to possible damage or breakage.

There is a possibility that even with proper instrument care and attention to surgical technique, these microsurgical instruments can malfunction, break, or bend, necessitating further surgical intervention.

PRECAUTION

Inspect all instruments before each use and maintain in a clean sharp condition.

Inspection

Inspect instrument before each use. Pay careful attention to the footplate area and the crossbar. Visible damage, developing cracks, bent components, or dullness indicate the instrument needs repair and must not be used.

Reprocessing Instructions

Refer to Symmetry Surgical Instructions for Use #204233 for updated information regarding reprocessing the instrument.

SEE CAUTIONS BELOW.

Copies of this instruction sheet are available by requesting IFU #204233 from Symmetry Surgical Customer Service at:

Telephone: 800 251 3000
Fax: +1 615 964 5566

E-mail:
ssi-customerservice@symmetrysurgical.com

CAUTION: Inspect instrument closely after ultrasonic cleaning for signs of disassembly such as loose screws and pins.

Service and Repair

For service and repair outside the United States, contact your local Symmetry Surgical representative.

Inside the United States, send instruments for service and repair to:

Symmetry Surgical
3034 Owen Drive,
Antioch, TN 37013 USA

Always include a repair purchase order number and a written description of the problem.

Warranty

Symmetry Surgical warrants that this medical device is free from defects in both materials and workmanship. **Any other express or implied warranties, including warranties of merchantability or fitness, are hereby disclaimed. Suitability for use of this medical device for any particular surgical procedure should be determined by the user in conformance with the manufacturer's instructions for use. There are no warranties that extend beyond the description on the face hereof.**

® MICROSECT and OPTI-LENGTH are registered trademarks of Symmetry Surgical

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