


Instructions for Use: REF# AAR100

1. Remove from packaging and place the positioner at the cephalad end of the O.R. Table.
2. Position the patient so that the top of the head is near the tallest portion of the positioner.
3. Remove any bottom layers necessary to achieve proper alignment of the patient's cervical spine.
4. Ensure that the patient's ear that is in contact with the positioner is oriented in the center of the ear hole.
5. Ensure that the patient's dependent eye is unobstructed, visible and oriented in the center of the eye channel.
6. Discard after use.

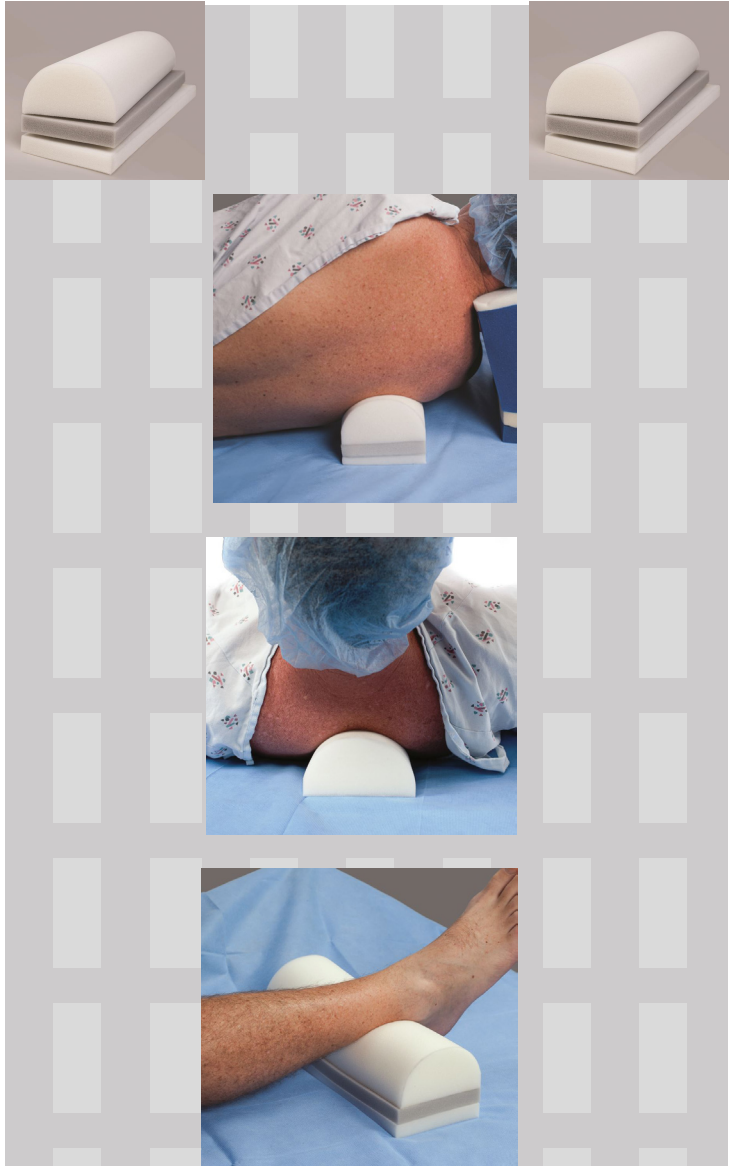

Not made with natural rubber latex



Manufactured For:


**PRIME
MEDICAL**
 1410 E Weisgarber
 Knoxville, TN 37909
 865-776-8994



QNET BV
 Kantstraat 19
 NL-5076NP Haaren
 The Netherlands