

## Instructions for Use: REF# AAR100

1. Remove from packaging and place the positioner on the O.R. Table at the desired location according to your positioning needs.
2. The AAR100 has two bottom removable layers that allow the positioner height to be adjusted to accommodate the patient and the procedure .
  - a. Remove any bottom layers necessary to achieve proper positioner height.
3. Discard after use.


## Providing a Better Way

The design, density and texture of the foam provide stability and support, while the removable bottom layers allow the clinician to adapt the AAR100 to the patient and procedure. Use of the AAR100 eliminates the need for rolled towels, blankets, I.V. bags, etc., which are not positioning devices, or gel rolls which require cleaning between surgical cases.

Because the AAR100 is single-patient-use, the risk of cross-contamination from reusable positioners is eliminated. So, too, is the risk of a dermatological reaction to a cleaning solution residue. By removing the need for a cleaning protocol, patient safety is enhanced and OR time is saved.

The AAR100 is your consistent, safe and universal surgical positioning solution for a wide variety of surgical procedures.

Not made with natural rubber latex

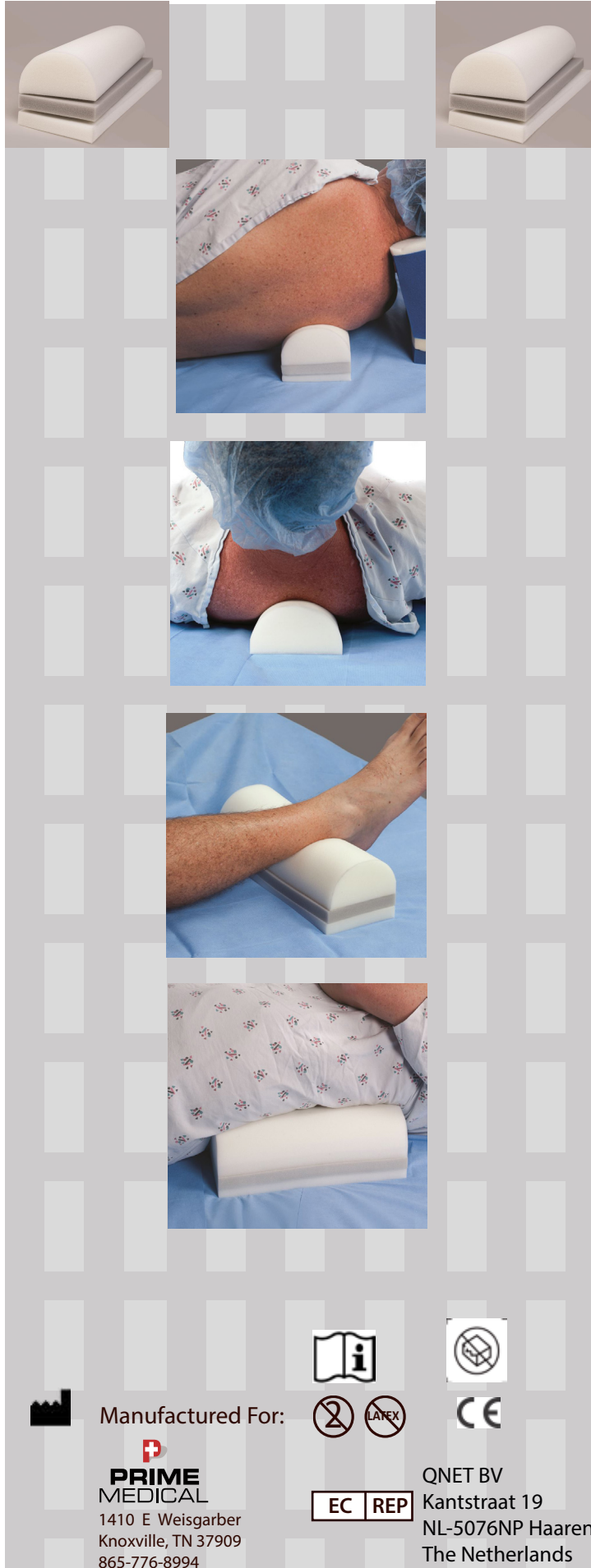

Manufactured For:



**PRIME  
MEDICAL**  
1410 E Weisgarber  
Knoxville, TN 37909  
865-776-8994



EC REP



QNET BV  
Kantstraat 19  
NL-5076NP Haaren  
The Netherlands