## INSTRUCTIONS FOR USE: Lap PakOR™

Single-piece bowel packing retractor, x-ray detectable

EN

## INDICATIONS FOR USE

The Lap PakOR is an x-ray detectable, translucent silicone rubber non self-retaining retractor used to assist surgeons in bowel packing. A single-use device, it is used in a patient undergoing an abdominal/pelvic procedure in lieu of, or in addition to, standard cotton towels and laparotomy pads.

The Lap PakOR assists in packing and restraining the bowels in the upper abdominal cavity to leave the surgical field clear of obstructions. The Lap PakOR serves to temporarily restrict the motion of the intestines by preventing them from entering the surgical field.

## CONTRAINDICATIONS

The Lap PakOR is not intended for use except as indicated. The device should not be used on injured tissue.

## CAUTIONS / WARNINGS


- Only physicians familiar with general surgical procedures (incl. gynecological, urological, colorectal and trauma surgery) should use the Lap PakOR.
- Device is designed for short term use but not for more than 24 hours.
- Ensure the Lap PakOR is not retained in the abdominal cavity post procedure. Visually and physically inspect the cavity to ensure that the Lap PakOR has been removed prior to closing the surgical site.
- The device is intended for single use only. The design of this device may not perform as intended by the manufacturer if it is reused. The manufacturer cannot guarantee the performance, safety and reliability of a reprocessed device.

NOTE: Retractor blades are not provided with the Lap PakOR.

STORAGE CONDITIONS: Store in a dry, dark place.

## INSTRUCTIONS FOR USE

- Once abdominal incision has been made and retracted in accordance with established protocol, insert the Lap PakOR into the abdominal cavity with the unmarked concave side cranial, facing the intestines. The Lap PakOR may be rolled or folded to adequately fit through the incision and then be expanded once in place.
- Position the Lap PakOR so that the small lower flap is posterior to the intestines, the side flaps are lateral to the intestines, and the top flaps are anterior to the intestines. The thickest section of the Lap PakOR will then be mostly in line with the transverse plane.
- Carefully move the intestines towards the thoracic cavity to clear the surgical field using gentle pressure against the back and/or sides of the Lap PakOR device to scoop the intestines.
- Use retractor blades to secure the Lap PakOR in place. Position two retractor blades vertically, flat against the thickest section of the Lap PakOR body.
- Secure the retractor blades to the table or a frame/ring in accordance with established protocol. Perform the procedure as planned.
- Reposition or ensure the Lap PakOR is secure as necessary. Monitor patient as per standard protocol.
- Once the procedure is complete, remove the retractor blades. Remove the Lap PakOR from the abdominal cavity. Discard after single use in approved medical waste container in accordance with institutional policy.
- Examine the organs and other organs for signs of potential damage and treat in accordance with the established protocol. Complete the procedure as per established protocol.

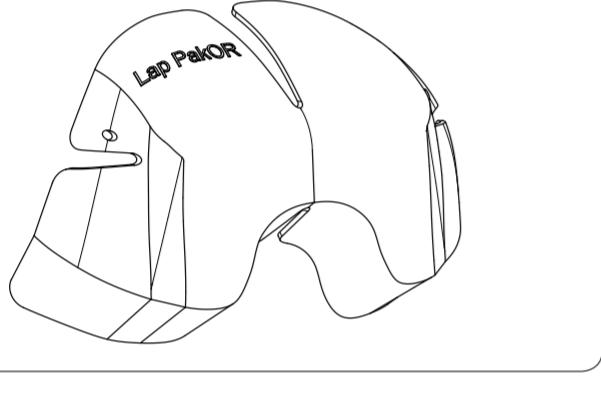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Caution, consult accompanying documents.

Do not use if package is damaged.

Consult Instructions for Use.

Do not reuse.

Keep away from sunlight.

Do not re-sterilize.

Keep dry.

Not made with natural rubber latex.

Caution: US Federal Law restricts this device to sale by or on the order of a physician.

Product conforms with essential requirements in the European Medical Devices Directive 93/42/EEC.

Distributed exclusively in USA by Symmetry Surgical, 3034 Owen Drive, Antioch, TN 37013, USA.

Telephone: US & Canada +1 800-251-3000 (Monday - Friday, 7:00AM - 5:30PM CST)

Email: customerservice@symmetriesurgical.com

The O R Company Pty Ltd  
1/32 Silwood Rise, Carrum Downs, VIC, 3201, Australia

OBELOS SA, Bd. Général Wahis 53, 1030 Brussels, BELGIUM

Tel: +32 2732 5954 Fax: +32 2732 6003 Email: mail@obels.net

Issue date: 21-12-2020

PM\_W18\_101 V3