**Symmetry Surgical® Air Powered Sternum Saw
Sterilizing Case
Symmetry Surgical I.M.A. Instrument Case
Symmetry Surgical Posterior Lumbar Interbody Fusion
(PLIF) II Sterilizing Case
Symmetry Surgical Power System Sterilizing Case
CROCKARD™ Transoral Instrument Set Sterilizing Case
MICROCARE® Microsurgical Instrument Case
MICROCARE Instrument Case with Flex Insert
MICROCARE Universal Bayonet Case**

Symmetry Surgical Inc.
3034 Owen Drive
Antioch, TN 37013 USA
☎ 1-800-251-3000
☎ Fax: 1-615-964-5566
www.symmetrysurgical.com

Symmetry Surgical GmbH
Maybachstraße 10
78532 Tuttlingen, Germany
☎ +49 7461 96490
☎ Fax: +49 7461 77921

LCN 205022-001/F
© 2005–2015 Symmetry Surgical
♻️ Revised 06/15

ENGLISH

IMPORTANT INFORMATION

Please Read Before Use

Symmetry Surgical® Air Powered Sternum Saw Sterilizing Case

Symmetry Surgical I.M.A. Instrument Case

Symmetry Surgical Posterior Lumbar Interbody Fusion (PLIF) II Sterilizing Case

Symmetry Surgical Power System Sterilizing Case

CROCKARD™ Transoral Instrument Set Sterilizing Case

MICROCARE® Microsurgical Instrument Case

MICROCARE Instrument Case with Flex Insert

MICROCARE Universal Bayonet Case

Brief Description

The instrument cases are designed for storage of surgical instruments during handling, sterilization, and use.

WARNINGS

For the Symmetry Surgical I.M.A. Instrument Case **only**: Always align the holes in the base of the flexible mat with the drainage holes in the instrument case. Failure to do so may preclude sterilization and prevent proper drainage from the case. Additional holes may be cut or punched in the mat if needed.

Precautions

Do not stack items on the case during sterilization.

Do not immerse the case in instrument lubricant. If instrument lubricant is used on instruments, remove excess lubricant before placing instruments in case.

For Symmetry Surgical I.M.A. Instrument, MICROCARE® Microsurgical Instrument, and MICROCARE Instrument Cases:

Do not use phenol solutions on the case.

Curved instrument tips should point up toward the lid.

For Symmetry Surgical I.M.A. Instrument Case:

The blade of the diamond knife should always be retracted before placing the knife in the case.

For MICROCARE Microsurgical Instrument Case:

Place instruments in tray so the tips point toward the silicone bumpers.

Product Description

CASE NAME	MATERIALS	SPECIAL FEATURES	STERILIZATION METHOD	FOR USE WITH
AIR POWERED STERNUM SAW STERILIZING CASE	Stainless Steel		Ethylene Oxide or steam	Air Powered Sternum Saw Set
Symmetry Surgical I.M.A. INSTRUMENT CASE	ULTEM® base with a polyethersulfone cover	Removable tray, angled design for use on Mayo stand	Ethylene Oxide or steam	I.M.A. Instruments
Symmetry Surgical POSTERIOR LUMBAR INTERBODY FUSION (PLIF) II STERILIZING CASE	Stainless Steel		Ethylene Oxide or steam	Symmetry Surgical PLIF II Kit
Symmetry Surgical POWER SYSTEM STERILIZING CASE	Stainless Steel		Ethylene Oxide or steam	Symmetry Surgical Power System Unitized Air Craniotome Set OR Symmetry Surgical Power System Modular Air Craniotome Set
CROCKARD™ TRANSORAL INSTRUMENT SET STERILIZING CASE	ULTEM 1000 Resin	Removable Tray	Steam	CROCKARD Transoral Instrument Set
MICROCARE® INSTRUMENT CASE WITH FLEX-INSERT	Rigid polyetherimide		Ethylene Oxide or steam	MICRA Titanium Instruments
MICROCARE® MICROSURGICAL INSTRUMENT CASE	Rigid polyetherimide	Removable tray, angled design for use on Mayo stand	Ethylene Oxide, steam, or flash	THE FOLLOWING MICROSURGICAL INSTRUMENT LINES: <ul style="list-style-type: none"> • MALIS® • Rhoton • MICRO VASC® • Symmetry Surgical Stainless Steel • E•SERIES®
MICROCARE® UNIVERSAL BAYONET CASE	ULTEM 1000 Resin	Removable Tray	Steam	Bayonet Instruments

Sterility

Refer to Symmetry Surgical Instructions for Use #204233 for updated information regarding reprocessing these cases.

Copies of this instruction sheet are available by requesting IFU #204233 from Symmetry Surgical Customer Service at:

Telephone: 800 251 3000
Fax: +1 615 964 5566
E-mail: ssi-customerservice@symmetrysurgical.com

Service and Repair

For service or repair outside the United States, contact your local Symmetry Surgical representative. Inside the United States, send instruments for service or repair to:

Symmetry Surgical
3034 Owen Drive,
Antioch, TN 37013 USA

Always include a repair purchase order number and a written description of the problem.

Warranty

Symmetry Surgical warrants that the Symmetry Surgical I.M.A. Instrument Case is free from defects in both materials and workmanship for one (1) year from the date of purchase. All other cases are warranted to be free from defects in both materials and workmanship. **Any other express or implied warranties, including warranties of merchantability or fitness, are hereby disclaimed. Suitability for use of this medical device for any particular surgical procedure should be determined by the user in conformance with the manufacturer's instructions for use. There are no warranties that extend beyond the description on the face hereof.**

MICROCARE®, MICRO VASC, and E•SERIES are registered trademarks of Symmetry Surgical
CROCKARD™ is a registered trademark of Hillway Surgical, Ltd.
MALIS® is a registered trademark of Synergetics USA, Inc.
ULTEM® is a registered trademark of General Electric Company

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