Hudson Cranial Drill Set

Symmetry Surgical Inc.
3034 Owen Drive,
Antioch, TN 37013 USA
☎ 1-800-251-3000
Fax: 1-615-964-5566
www.symmetrysurgical.com

Symmetry Surgical GmbH
Maybachstraße 10
78532 Tuttlingen, Germany
☎ + 49 7461 96490
Fax: + 49 7461 77921

LCN 260-1075-85/L

© 2002–2015 Symmetry Surgical

♻ Revised 06/15

ENGLISH

IMPORTANT INFORMATION

Please Read Before Use

Hudson Cranial Drill Set

Rx Only

Description

The Hudson Cranial Drill Set was designed for use in craniotomy procedures. The set includes four different bur sizes, a four inch (10.2 cm) cerebellar extension, and a ten inch (25.4 cm) Hudson Brace with snap-lock chuck. The extension accepts all four bur sizes for access to the cerebellar region. The burs are designed to fit on both the Hudson Brace and the cerebellar extension. The drill set is supplied nonsterile and must be sterilized before use, including initial use.

Hudson Cranial Drill Set, catalog no. 26-1075, includes (see Figure 1):

Instrument	Catalog No.
A. Bur, 9 mm	15-1005
B. Bur, 14 mm	15-1006
C. Bur, 16 mm	15-1007
D. Bur, 22 mm	15-1008
E. Hudson Cerebellar Extension, 4 in. (10.2 cm)	80-1002
F. Hudson Brace, 10 in. (25.4 cm)	26-1041
G. Sterilizing Case	21-1002

The above instruments are assembled in a convenient sterilizing-carrying case.

PRECAUTIONS

Bone cutting instruments have a tendency to become dull after repeated use and should be sharpened on a regular basis.

Cleaning

Clean the instruments before initial use, and after each use. Disassemble and scrub the instruments thoroughly using a soft brush and

a mild detergent. Remove all traces of blood and debris. Make sure all moveable parts are cleaned thoroughly to prevent debris from interfering with movement.

Lubrication

Lubricate the instruments after each procedure. Place one drop of water soluble lubricant on each moveable part where contact with other parts occurs. PRESERVE® Concentrate (catalog no. 43-1033) is recommended for instruments to be steam sterilized.

Sterilization

The instruments may be steam or gas sterilized. Refer to the sterilizer manufacturer's instructions for correct times, temperatures and pressure settings.

Service and Repair

For service or repair outside the United States, contact your local Symmetry Surgical representative.

Inside the United States, send instruments for service or repair to:

Symmetry Surgical
3034 Owen Drive,
Antioch, TN 37013 USA

Always include a repair purchase order number and a written description of the problem.

Warranty

Symmetry Surgical warrants that this medical device is free from defects in both materials and workmanship. **Any other express or implied warranties, including warranties of merchantability or fitness, are hereby disclaimed. Suitability for use of this medical device for any particular surgical procedure should be determined by the user in conformance with the manufacturer's instructions for use. There are no warranties that extend beyond the description on the face hereof.**

PRESERVE® is a registered trademark of Symmetry Surgical

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® PRESERVE is a registered trademark of Symmetry Surgical