Arm Boards with Soft Straps Aspen Surgical Non-Sterile

Rx ONLY MD

Aspen Surgical Products, Inc.
6945 Southbelt Drive SE
Caledonia, MI 49316 USA
Phone +1 616-698-7100
Toll-Free 888-364-7004
aspensurgical.com

EC REP

Emergo Europe
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands

For Country of Origin See Product Label

English

Intended Use

Intended to support and maintain the position of an anesthetized patient during surgery.

Intended User/Patient Target Groups

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

Instruction for Use:

- Position Arm Board Pads between equipment and the patient's arm. Adjust the pad to the desired length by using the hook-and-loop on the bottom of the pad. This will also create a natural rest position for the hand.
- Use the Soft Straps to secure the patient's arm to the Arm Board.
- Device is intended for limited contact duration, less than or equal to 8 hours, less if damaged.
- Discard after use.

Precautions:

- Ensure pad/cover is secure in the positioning device and that it stays in place while the patient is on the equipment.
- Take care when positioning patient to avoid any pinching or pressure on the skin.
- Use with approved positioning device.
- Do not sterilize.

Warning

Reuse of device could result in infection/contamination and/or device failure which could lead to patient harm.

Disposal

Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional and/or local laws and regulations.

Notice to Users and/or Patients in EU

Any serious incident that has occurred in relation to the device should be reported to Aspen Surgical Inc., the governing body, and the Competent authority of the member state in which the incident occurred.

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Emergo Consulting (UK) Limited