Surgidyne® Closed Wound Drainage Systems

Surgidyne Closed Wound Drainage Systems are sterile, portable closed systems designed to evacuate post-operative wound drainage and promote healing. Systems may include:

- S-Vac Silicone Bulb Evacuator
- S-Vac PVC Bulb Evacuator
- Saber® Infection Control Bulb Evacuator with 3C™ Collection Bag
- S-Vac Fluted Channel Wound Drains (Silicone)
- S-Vac Perforated Wound Drains (Silicone)
- Accessories

Aspen Surgical Products Inc.
6945 Southbelt Drive SE
Caledonia, MI 49316
Phone 616-698-7100
Toll-Free 888-364-7004
aspensurgical.com


Made in USA

R_x ONLY

STERILE EO

70090-021 | Rev 1 | 15SEPT2020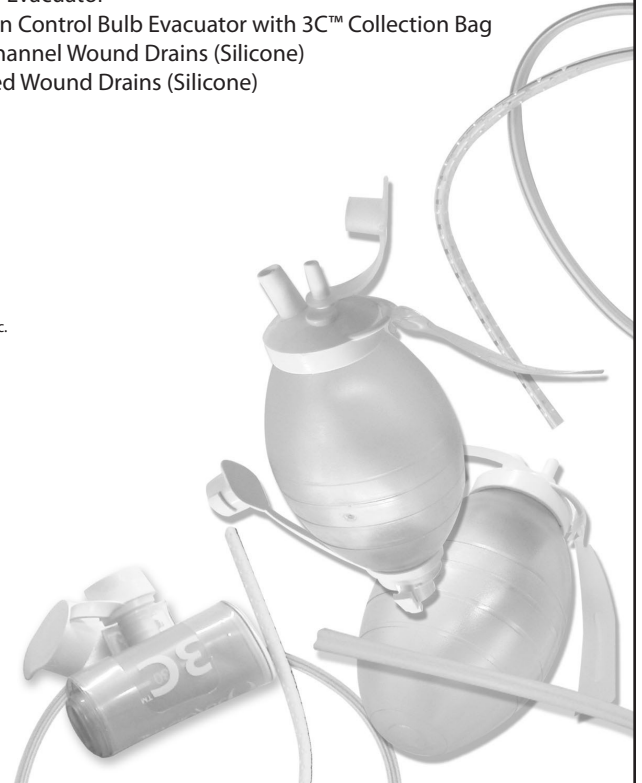

PRECAUTION:

- Drains should be carefully placed and removed by hand. Handling drains with instruments may cause them to tear and subsequently break.
- Drains should be visually checked to remove any kinks or bends before closure.
- To create and maintain effective suction/vacuum, an air-tight seal must be achieved and maintained between the wound, drain and collection system. All connections must be dry when assembling and be engaged as far onto the connectors as possible to maintain suction/vacuum.
- Never suture or cut through the drain tubing that is exposed outside of the patient exit site.
- Activate suction/vacuum as soon as possible following drain placement to maintain drain patency.
- If a drain becomes occluded it may be necessary to irrigate or aspirate to resume drainage.
- If the Adaptor tube is kinked, suction may be lost.
- Device is single use only. Reuse of this device may lead to patient cross contamination and/or device failure.
- Device is intended for prolonged contact duration, 24 hours up to 30 days.
- Contraindication: Not intended for routine abdominal drainage after hepatic resection in patients with chronic liver diseases.

A. DRAIN PLACEMENT

Step 1: Physician places the drain.

Step 2: Ensure that perforations or channels are completely inside the wound area.

Step 3: Suturing or taping is recommended around the drain tubing at the exit site to help prevent accidental drain displacement.


B. ACTIVATING/EMPTYING THE S-VAC BULB EVACUATOR

Step 1: Verify patency of S-Vac Bulb Evacuators


- Open emptying port cap.
- Compress the bulb, forcing all air out of the bulb.
- Seal both top ports on S-Vac bulb evacuator using the emptying port cap and your finger over the drain inlet port.
- Release hand pressure on the bulb
- The bulb evacuator should not re-inflate with air. If it does re-inflate the bulb should not be used.
- Remove finger from the drain inlet port. Bulb should re-inflate.

Step 2: Connect Wound Drain

- Connect Drain to drain inlet port of S-Vac Bulb Evacuator.
- Push tubing firmly onto connector until it reaches the base of the port. If using two drains, a Y connector must be used such as Surgidyne Cat.# 450121 or Bifurcated Y Connector.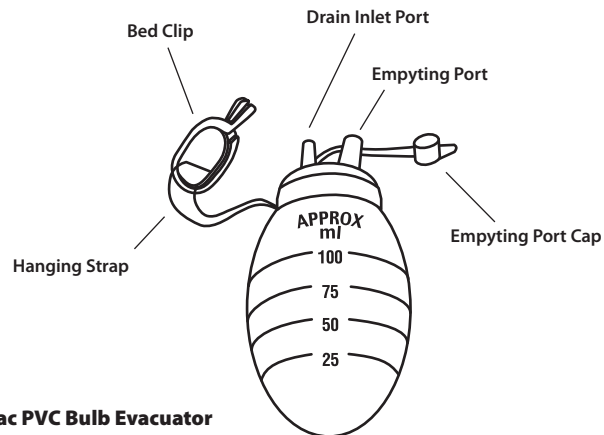

S-Vac PVC Bulb Evacuator

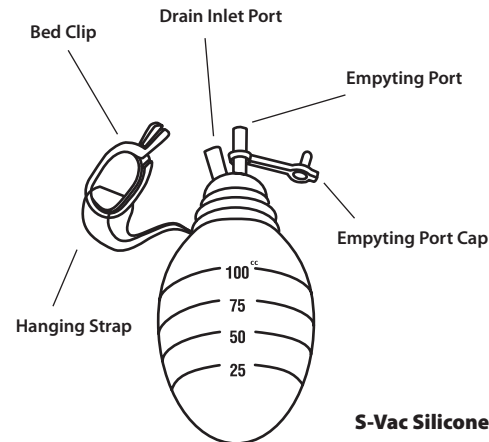S-Vac Silicone Bulb Evacuator

Step 3: Activating S-Vac Bulb Evacuator

- Following wound drain connection, with the emptying port uncapped, activate bulb suction by compressing the bulb evacuator to expel all air.
- Maintain bulb evacuator compression and cap the emptying port.
- Release bulb compression to initiate bulb suction.

Step 4: Emptying The S-Vac Bulb Evacuator

- Uncap Emptying Port.
- Invert bulb evacuator above collection receptacle.
- Gently squeeze the center of the bulb to discharge the collected fluid.
- Cap Emptying Port of collapsed bulb to reactivate suction/vacuum.

To create and maintain effective suction/vacuum an air-tight seal must be achieved and maintained between the wound, drain and collection system. All connections must be dry when assembling and be engaged as far onto the connectors as possible to maintain suction/vacuum.

ACTIVATING THE SABER BULB EVACUATOR WITH 3C™ COLLECTION SYSTEM

Step 1: Verify patency of Saber Bulb Evacuator

- Open 3C emptying port, on the bottom of the bulb evacuator and cap the top emptying port.
- Compress the bulb, forcing all air out of the bulb.
- Seal both top ports on S-Vac bulb evacuator using the emptying port cap and your finger over the drain inlet port.
- Release hand pressure on the bulb.
- The bulb evacuator should not re-inflate with air. If it does re-inflate the bulb should not be used.
- Remove finger from drain inlet port. Bulb should re-inflate.

Step 2: Connect Wound Drain

- Connect Drain to drain inlet port of Saber Bulb Evacuator.
- Push tubing firmly onto connector until it reaches the base of the port. If using two drains, a Y connector must be used such as Surgidyne Cat.# 450121 or Bifurcated Y connector.

Step 3: Activating Saber Bulb Evacuator

- Following wound drain connection, with the emptying port uncapped, activate bulb suction by compressing the bulb evacuator to expel all air.
- Maintain bulb evacuator compression and cap top emptying port, and either cap bottom emptying port or attach the optional 3C collection system.
- Release bulb compression to initiate bulb suction.

Step 4: Attaching the Optional 3C Collection System

- Maintain bulb compression and open 3C emptying port.
- Compress ends of latches on 3C port and firmly seat connector of 3C Collection System until latches engage.

EMPTYING THE SABER WITH THE 3C SYSTEM AND RE-ACTIVATE SUCTION

Step 1: Remove the 3C Collection System Retention Sleeve and unroll 3C Collection System.


Step 2: Squeeze the center of the SABER to simultaneously empty it into the 3C Collection System and re-activate suction.

(DO NOT open the emptying port on the top of the bulb.)

Step 3: To remove 3C Collection System, hold bulb with bottom emptying port at an upward angle (45 degrees or more) and squeeze the cap latches on the emptying port with fingers placed at the ends of the latch arms. With the other hand twist the 3C Connector and remove unit. Seal 3C Collection System with closure cap and set aside to discard.

Step 4: Place new 3C Collection System on Emptying Bottom Port or cap Port without 3C Collection System.

Step 5: Dispose of 3C Collection System according to hospital procedure for discarding potentially contaminated waste.

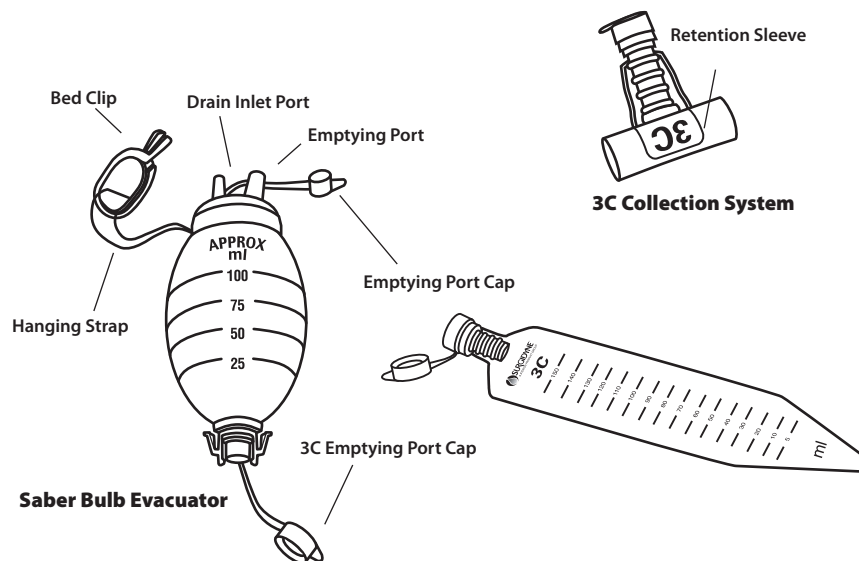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