

 symmetry surgical® | **Bovie**®

PLASTIC SURGERY PRODUCTS



THE NEXT GENERATION OF ELECTROSURGERY GENERATORS



A3350 OR | PRO 300

Feature: The dual monopolar ports allow two surgeons to work simultaneously in the spray coagulation setting.
Benefit: Two surgeons can work in separate areas, allowing the procedure to be completed faster.

Feature: The CUT II mode has a low peak to peak voltage (Vp-p) of 750Vp-p. This allows for less tissue damage when performing invasive surgical procedures using any procedural scope.
Benefit: When performing a laparoscopic cutting procedure and cauterizing tissue, the CUT II mode causes less damage to the tissue, allowing for faster healing time. This mode may help prevent direct and capacitance coupling.

Feature: The Auto Bipolar setting allows the surgeon to activate the bipolar forceps without a foot pedal.
Benefit: The surgeon does not have to look down to locate the foot pedal, allowing for quicker activation of the bipolar forcep, saving time and motion during the procedure.

Feature: The Bovie® Bipolar (BBp mode) has more power than the Macro, Micro, and Standard bipolar settings.
Benefit: This mode provides enough power for the most demanding surgical procedures.

Feature: Gentle Coag Mode
Benefit: This mode provides gentler coagulation when a Gold Probe™* is used during a colonoscopy to prevent perforation of the intestine.

Feature: Split Pad Bovie NEM Technology
Benefit: Complete patient safety to prevent burns during procedures



A2350 Surgi-Center | PRO

Feature: The dual monopolar ports allow two surgeons to work simultaneously in the spray coagulation setting.
Benefit: Two surgeons can work in separate areas, allowing the procedure to be completed faster.

Feature: The CUT II mode has a low peak to peak voltage (Vp-p) of 750Vp-p. This allows for less tissue damage when performing invasive surgical procedures using any procedural scope.
Benefit: When performing a laparoscopic cutting procedure and cauterizing tissue, the CUT II mode causes less damage to the tissue, allowing for faster healing time. This mode may help prevent direct and capacitance coupling.

Feature: The surgeon can move smoothly from monopolar to bipolar.
Benefit: Saves the surgeon time and motion during the procedures.

Feature: Four blend settings
Benefit: Allows the surgeon to customize the amount of coag included in the cut setting

Feature: Split Pad Bovie NEM Technology
Benefit: Complete patient safety to prevent burns during procedures



A1250S - Specialist | PRO

Feature: Affordable solution for most in-office electrosurgery procedures.
Benefit: The surgeon receives a much faster return on their investment.

Feature: Cut and coag activation is on the handpiece.
Benefit: The surgeon can move seamlessly from cut to coag without leaving the sterile field.

Feature: The push-button controls allow easier use of the generator.
Benefit: The push-button front panel saves time and motion in adjusting the generator. The smoothly embossed push-button front panel allows the staff to clean the generator more quickly since there are no knobs.

Feature: Split Pad Bovie NEM Technology
Benefit: Complete patient safety to prevent burns during procedures, just like our larger units in the Operating Room

Perfect option for MOHS procedures



UNSURPASSED 4-YEAR WARRANTY ON ALL BOVIE ELECTROSURGERY GENERATORS!

*Gold Probe is a product of Boston Scientific



GENERATOR COMPARISON

FEATURES	A3350	A2350	A1250S
Cut (# of modes)	300 W (2)	200 W (2)	120 W (1)
Blend (# of steps)	200 W (4)	200 W (4)	90 W (1)
Pinpoint Coag	120 W	120 W	80 W (Coag)
Fulguration/Spray Coag	120 W	120 W	40 W
Gentle Coag	120 W	120 W	N/A
Macro Bipolar	150 W	80 W	N/A
Standard Bipolar	50 W	50 W	30 W
Micro Bipolar	80 W	80 W	N/A
Bovie Bipolar	225 W	N/A	N/A
Pad Alarm	YES	YES	YES
Pad Sensing (tissue impedance)	YES	YES	YES
Presets (# of presets)	(10)	(10)	N/A




FOOTCONTROL / HANDPIECE ACCESSORIES	A3350	A2350	A1250S
Foot Control	YES	YES	YES
Standard 3-Pin	YES	YES	YES
ESPR2 Reusable Pencil	YES	YES	YES
Two Pencil Configuration	YES	YES	N/A
US, Canada	YES	YES	YES
CE	YES	YES	YES

ILLUMINATING THE PATH TO BETTER HEALTHCARE

MEDICAL LIGHTING

Bovie® offers energy efficient LED medical lights to provide 50,000 hours of cool, clear visibility that are ideal for patient exams or small surgical procedure rooms. Control your lighting options with one hand. Bovie medical lighting provides a multi-dimming and illumination field that is controlled with a sterilizable handle keeping you in a sterile field.

Bovie continues to prioritize safety with the UV24 Air Purification systems. These units function as ceiling lights while also providing 24/7 protection to your patients and staff against airborne bacteria, viruses and mold in all areas of your facility.






 <p>MI 750</p>	 <p>MI 1000</p>
 <p>System Two LED</p>	 <p>UV24*</p>

*Incorporates office lighting options only.

RECOMMENDED COMPLEMENTARY PRODUCTS

Smoke Evacuation System

Bovie® Smoke Shark® II is designed to be easy to use and is effective for controlling surgical smoke generated from the use of electro-surgical, laser, and ultrasonic energy products.

 <p>SE02 Smoke Shark® II unit with pneumatic footswitch (FSSEP)</p>	 <p>SEPAT Smoke Evacuation Pencil Adaptor and Tubing, 3/8" x 4' Hose with 7/8" to 6' Tube</p>	 <p>SERS2 Bovie® Smart Shark Remote Activator Switch</p>
 <p>SS95 SharkSkin electrosurgery pencil adaptor with 3/8" (10mm) x 10' tubing</p>	 <p>ISEP1000 Bovie® Orca™ Smoke Evacuation Pencil, 10ft Tubing, Sterile</p>	

RECOMMENDED COMPLEMENTARY PRODUCTS (continued)

Mobile Carts



ESMS2
Mobile stand for A1250S




BV-IDS-CS2
Mobile stand for A3350, A2350 and A1250S

Pencils




ESP1
Sterile push-button pencil with blade electrode



ESP6
Disposable rocker-switch pencil with blade

Return Electrodes



ESRE
Split adult return electrode



ESREC
Split adult return electrode w/ 2.8m cable

Bipolar footswitch accessories



A824
Reusable bipolar forceps, 4.5", curved, micro (0.5mm) tips



A827V
Bipolar forceps cord, fits A3350, A2350, A1250S, A952, A942 and DERM 102



A1255A
Adaptor for connecting footswitch pencil to A3350, A2350 and A1250S



BV-1254B
Bipolar footswitch for A3350 and A2350






BV-1253B
Monopolar footswitch for A3350, A2350 and A1250S










RECOMMENDED COMPLEMENTARY PRODUCTS (continued)



Microdissection Needle Electrodes*

 <p>97130BX Microdissection Needle Electrode, Tungsten Wire, Straight, Insulated, Ultra Sharp Tip, 3cm</p>	 <p>97133BX Microdissection Needle Electrode, Tungsten Wire, Straight, Insulated, Ultra Sharp Tip, 6.4cm</p>	 <p>97150BX Microdissection Needle Electrode, Tungsten Wire, 10mm Angled Shaft, Ultra Sharp Tip, Insulated, 5cm</p>
--	--	---

Specialty Electrodes*






 <p>ES60 SuperCut™ tungsten needle, super fine, 3cm, sterile</p>	 <p>ES61 SuperCut™ tungsten needle, modified super fine, 3cm, sterile</p>	 <p>ES62 SuperCut™ tungsten needle, super fine, 4.5cm, sterile</p>
 <p>ES63 SuperCut™ tungsten needle, modified super fine, 4.5cm, sterile</p>	 <p>ES61HS SuperCut™ tungsten needle, modified fine, with Heatshrink, sterile</p>	

Resistick II™ Coated Electrodes*

 <p>ES58T Blade electrode, 2.5" (6.4cm)</p>	 <p>ES39T Extended modified blade electrode, 6" (15.2cm)</p>	 <p>ES20T Standard 3/16" (4.8mm) ball electrode</p>	 <p>ES02T Standard needle electrode, 2.75" (7cm)</p>
---	--	--	--

*More electrode styles available.

SYMMETRY SURGICAL PLASTIC SURGERY INSTRUMENTATION

 <p>15-4512 Symmetry® Forceps, Halsted Mosquito, Curved, 5 in (122 mm)</p>	 <p>20-0508 Symmetry® Retractor, Senn, Double-Ended, Sharp, 6-1/4 in</p>	 <p>32-500 Symmetry® Forceps, Adson Dressing, Tungsten Carbide, 4-3/4 in</p>
 <p>60-1856 Symmetry® Scissors, Strabismus, Straight, Ribbon Handles, 4-1/4 in</p>	 <p>60-1924 Symmetry® Needle Holder, Castroviejo, Straight, w/Lock, 12 mm, 5-1/2 in</p>	

*Products featured.

Item Number	Description	Quantity
01-2703	Symmetry® Knife Handle, #3, Standard, 5 in, 127 mm	1/each
01-2804	Symmetry® Knife Handle, #4, Standard, 5-1/4 in, 133 mm	1/each
01-2814	Symmetry® Knife Handle, #4L, Long, 8-3/8 in, 212 mm	1/each
02-2011	Symmetry® Scissors, Iris Standard Pattern, Straight, 4-1/2 in	1/each
02-2111	Symmetry® Scissors, Iris Standard Pattern, Curved, 4-1/2 in	1/each
02-5002	Symmetry® Scissors, Tenotomy, Straight, Sharp, 4-1/4 in	1/each
02-5003	Symmetry® Scissors, Tenotomy, Curved, Sharp, 4-1/8 in	1/each
03-0415	Symmetry® Scissors, Operating, Straight, Sharp/Sharp, 6 in	1/each
03-0613	Symmetry® Scissors, Operating, Straight, Sharp/Blunt, 5-1/8 in	1/each
03-5017	Symmetry® Scissors, Mayo, Straight, Beveled Blades, 6-3/4 in (173 mm)	1/each
03-5117	Symmetry® Scissors, Mayo, Curved, Beveled Blades, 6-3/4 in	1/each
04-3214	Symmetry® Scissors, Metzenbaum Dissecting, Straight, Standard Pattern, 5-3/4 in	1/each
04-3314	Symmetry® Scissors, Metzenbaum, Curved, Standard Pattern, 5-3/4 in	1/each
04-3318	Symmetry® Scissors, Metzenbaum, Curved, Standard Pattern, 7 in (178 mm)	1/each
06-0014	Symmetry® Forceps, Dressing, Serrated, 5-1/2 in	1/each
06-0020	Symmetry® Forceps, Dressing, Serrated, 8 in	1/each
06-0514	Symmetry® Forceps, Tissue, 1x2 Teeth, 5-3/4 in (140 mm)	1/each
06-0520	Symmetry® Forceps, Tissue, 1x2 Teeth, 8 in	1/each
06-2012	Symmetry® Forceps, Adson Dressing, Serrated Jaws, 4-3/4 in	1/each
06-2112	Symmetry® Forceps, Adson Tissue, 1x2 Teeth, 4-3/4 in	1/each
06-2113	Symmetry® Forceps, Adson Tissue, Serrated, 1x2 Teeth, 4-3/4 in	1/each
06-2512	Symmetry® Forceps, Adson-Brown Tissue, 4-3/4 in (121 mm), 7 x 7 Teeth	1/each
06-2590	Symmetry® Forceps, Callison Tying Tissue, 1.5 mm Teeth, Tying Platform, 1x2 Teeth, 4-7/8 in	1/each
06-8798	Forceps, Heaney Tissue, 2x3 Teeth, 5-1/2 in	1/each
09-8309	Symmetry® Suction Tube, Frazier, Medium Angle, 9 Fr, 4-1/3 in, 110 mm	1/each
10-1012	Symmetry® Needle Holder, Webster, Straight, Blunt, Smooth Jaws, 5-1/4 in, 133 mm	1/each
10-1212	Symmetry® Needle Holder, Derf, 5 in, 127 mm	1/each
10-1213	Symmetry® Needle Holder, Halsey, Serrated, 5-3/8 in, 127 mm	1/each
10-1813	Symmetry® Needle Holder, Mayo-Hegar, 5-1/4 in, 133 mm	1/each
10-1815	Symmetry® Needle Holder, Mayo-Hegar, 6-2/3 in, 160 mm	1/each

SYMMETRY SURGICAL PLASTIC SURGERY INSTRUMENTATION (continued)

Item Number	Description	Quantity
10-3121	Symmetry® Needle Holder, Heaney, 8-1/4 in, 209 mm	1/each
14-0609	Symmetry® Scissors, Spencer Stitch, 3-1/2 in	1/each
15-3209	Symmetry® Forceps, Hartman Baby Mosquito, Straight, 3-7/8 in	1/each
15-3309	Symmetry® Forceps, Hartman Baby Mosquito, Curved, 4 in	1/each
15-4412	Symmetry® Forceps, Halsted Mosquito, Straight, 5 in	1/each
15-4512*	Symmetry® Forceps, Halsted Mosquito, Curved, 5 in (122 mm)	1/each
15-7016	Symmetry® Forceps, Kelly Artery, Straight, 6-1/4 in	1/each
15-7114	Symmetry® Forceps, Kelly Artery, Curved, 5-1/2 in (140 mm)	1/each
15-7116	Symmetry® Forceps, Kelly Artery, Curved, 6-1/4 in	1/each
15-7316	Symmetry® Forceps, Crile Artery, Curved, 6-1/4 in	1/each
16-1016	Symmetry® Forceps, Rochester-Pean Hemostatic, Straight, 6-5/8 in	1/each
16-1116	Symmetry® Forceps, Rochester-Pean Hemostatic, Curved, 6-3/5 in	1/each
17-5508	Symmetry® Clamp, Backhaus Towel, 3-1/2 in (89 mm)	1/each
17-5513	Symmetry® Clamp, Backhaus Towel, 5-1/4 in (133 mm)	1/each
19-5024	Symmetry® Forceps, Foerster, Straight, Serrated, 9-3/4 in	1/each
19-5579	Classic® Clamp, Gemini, 5-1/2 in	1/each
20-0508*	Symmetry® Retractor, Senn, Double-Ended, Sharp, 6-1/4 in	1/each
20-5001	Symmetry® Hook, Joseph Skin, Single-Prong, 6.0 mm, 6-1/4 in, 158 mm	1/each
20-5005	Symmetry® Hook, Joseph Skin, Double-Prong, 5 mm Spread, 4.5 mm, 6-1/4 in, 158 mm	1/each
22-1418	Symmetry® Retractor, Parker, Large, 7 in, (Set of 2)	1/each
23-5534	Symmetry® Curette, Sims Uterine, Size 1, Sharp, Malleable Shaft, 11 in	1/each
23-7503	Symmetry® Retractor, Deaver, 1 in Wide, Standard Handle, 9-1/4 in	1/each
24-2211	Symmetry® Retractor, Weitlaner, Blunt, 2x3 Prong, 4-1/2 in	1/each
30-5561	Symmetry® Forceps, Bozeman Dressing, Curved, 260 mm	1/each
30-6556	Symmetry® Forceps, Bishop Harmon Iris, Delicate	1/each
32-110	Symmetry® Needle Holder, Baumgartner, Tungsten Carbide, 5 in	1/each
32-405	Symmetry® Needle Holder, Webster, Tungsten Carbide, Large Ring Handle, 4-3/4 in	1/each
32-430	Symmetry® Needle Holder, Crile-Wood, 6 in, 152 mm	1/each
32-500*	Symmetry® Forceps, Adson Dressing, Tungsten Carbide, 4-3/4 in	1/each
50-0215	Symmetry® Forceps, Allis Tissue, 4x5 Teeth, 6 in (152 mm)	1/each
50-0219	Symmetry® Forceps, Allis Tissue, 5x6 Teeth, 7-1/2 in	1/each
50-5540	Symmetry® Retractor, Jackson Vaginal, Small	1/each
50-5541	Symmetry® Retractor, Jackson Vaginal, Medium	1/each
50-5542	Symmetry® Retractor, Jackson Vaginal, Large	1/each
52-0610	Symmetry® Speculum, Graves Vaginal, 1-1/4 in x 4 in Blades, Medium	1/each
52-2793	Speculum, Auvard Vaginal, 1-3/4 in x 4 in Blades, 2.5 lbs	1/each
52-4001	Symmetry® Sound, Sims Uterine, Graduated In Centimeters, 13-1/4 in, 335 mm	1/each
52-4426	Symmetry® Forceps, Braun Tenaculum, Straight, 9-1/2 in	1/each
54-2555	Symmetry® Scissors, Shortbent Stitch	1/each
54-4030	Symmetry® Scissors, Operating, Straight, 5-1/2 in	1/each
60-1813	Symmetry® Scissors, Stitch, Curved, Delicate, Sharp, Large Ribbon Ring Handles w/Flat Shanks, 4 in	1/each
60-1856*	Symmetry® Scissors, Strabismus, Straight, Ribbon Handles, 4-1/4 in	1/each
60-1924*	Symmetry® Needle Holder, Castroviejo, Straight, w/Lock, 12 mm, 5-1/2 in	1/each
60-1925	Symmetry® Needle Holder, Castroviejo, Curved, w/Lock, 12 mm, 5-1/2 in	1/each
67-5507	Scissors, Ragnell, Curved, Beveled Blades, Dissecting Tips, 5 in	1/each
86-5013	Symmetry® Scissors, Stevens Tenotomy, Curved, Serrated, Sc, 4 in	1/each





| **Bovie**

3034 Owen Drive | Antioch, TN 37013 | 800.251.3000 (US) | +41 (0) 52 557 59 00
customerservice@symmetrysurgical.com | [symmetrysurgical.com](https://www.symmetrysurgical.com)
©2021 Symmetry Surgical Inc. All rights reserved.
BOVIE BROCHURE PLASTIC SURGERY 05.05.2022 v1