BOOKWALTER® Retractor Component

Symmetry Surgical Inc.
3034 Owen Dr
Antioch, TN 37013 USA
 1-800-251-3000
Fax: 1-615-964-5566

[EC] [REP]

Symmetry Surgical GmbH
Maybachstraße 10, 78532
Tuttlingen, Germany

+ 49 7461 96490
Fax: +49 7461 77921

(For class 2 products the
notified body reference number
is 0086)

LCN 500-4556-85/U
© 2002-2015 Symmetry Surgical
 Revised 10/15

ENGLISH

IMPORTANT INFORMATION

Please Read Before Use

BOOKWALTER® Retractor Component

Rx Only

Indications

This instrument has been designed for use with one of our BOOKWALTER Retractor systems providing multi-directional exposure for various surgical specialties. Refer to the instruction manual that provides complete operating instructions and is supplied with the BOOKWALTER Retractor system you will employ.

Description

Refer to the package label for a description of contents.

WARNINGS AND PRECAUTIONS

The retractor system can exert a great amount of force on the wound edges and on tissues. Release the retractor blades for short periods of time or shift the blades to avoid having unrelieved pressure on any one point.

If the table is to be tilted toward the side holding the post, it is important to place adequate padding between the table post and the patient. Pressure necrosis or a nerve palsy could result if the patient is pressed against the unpadded bar for a long period.

Precautions

As with any self-retaining retractor, do not exert undue retraction pressure on the iliopsoas muscle to prevent possible femoral nerve palsy.

If using monopolar electrosurgical equipment while using any of our BOOKWALTER Retractor systems, consult the basic rules

of usage concerning electrosurgery (see EN 60 601-2-2), as well as the manufacturer's instructions for use regarding specific warnings and conditions.

Be sure to electrically isolate the patient from (high frequency leakage currents to) the operating table, in accordance with the instructions provided by the manufacturer of the electrosurgical equipment.

Ensure that the patient does not come into contact with metal parts that are earthed or that have an appreciable capacitance to earth (for example: operating table, supports), in accordance with the instructions provided by the manufacturer of the electrosurgical equipment.

Avoid small-area contacts with objects having a low impedance to earth at high frequencies, as unwanted burns can result.

Adjusting the Rotating Ratchet Mechanism

The rotatable ratchet mechanism can be adjusted to one of 7 indexed positions along an arc of 60 degrees. The pressure of the ring on the interior of the ratchet mechanism maintains the selected position. To adjust the rotation of a retractor blade during the procedure, pull the ratchet mechanism slightly away from the ring; rotate the retractor blade to another indexed position as desired; replace the ratchet mechanism on the ring.

Reprocessing Instructions

Refer to Symmetry Surgical Instructions for Use #204233 for updated information regarding reprocessing the instrument.

Copies of this instruction sheet are available by requesting IFU #204233 from Symmetry Surgical Customer Service at:

Telephone: 800 251 3000
Fax: +1 615 964 5566
E-mail: customerservice@symmetrysurgical.com

Proper Care

Surgical instruments will corrode and deteriorate if exposed to harsh chemicals and solutions. Do not expose them to acids, harsh detergents, or prolonged exposure to saline soaks. Inspect all surgical instruments carefully before each use.

Care and Handling

Much has been written about the proper care and handling of hand-held surgical instruments. We would be pleased to send a copy of our brochure, "The Care and Handling of Surgical Instruments." Inside the United States, call 800 251 3000 or contact your local Symmetry Surgical sales representative. Outside the United States, contact your local Symmetry Surgical representative.

Warranty

Symmetry Surgical warrants that this medical device is free from defects in both materials and workmanship. **Any other express or implied warranties, including warranties of merchantability or fitness, are hereby disclaimed.** Suitability for use of this medical device for any particular surgical procedure should be determined by the user in conformance with the manufacturer's instructions for use. There are no warranties that extend beyond the description on the face hereof.

® BOOKWALTER is a registered trademark of Symmetry Surgical

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© 2002–2015 Symmetry Surgical Inc.