INSTRUCTIONS FOR USE: QuivOR™ Surgical Instrument Holder

EN

INDICATIONS FOR USE / CLINICAL BENEFITS

QuivOR™ is a surgical instrument holder, reducing unnecessary instrument exchanges between surgeon and scrub personnel in the operation room.

CONTRAINDICATIONS

This device is not intended for use except as indicated.

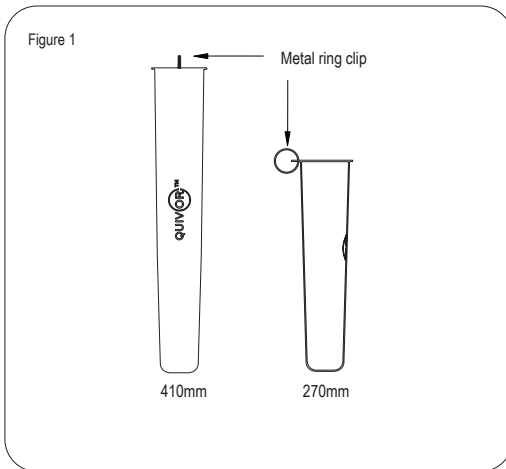
CAUTIONS / WARNINGS

- Choose the QuivOR™ size in relation to the surgical instruments used. Do not overload the instrument holder.
- Each QuivOR™ requires its own towel clamp.
- The device is intended for single use only. The design of this device may not perform as intended by the manufacturer if it is reused. The manufacturer cannot guarantee the performance, safety and reliability of a reprocessed device.

INSTRUCTIONS FOR USE

- Remove the QuivOR™ from the sterile packaging and place it on the sterile instrument table.
- Drape patient as required.
- Secure the QuivOR™ to the sterile patient drape using a towel, Kelly or artery clamp and the supplied 4mm key ring.
- Ensure the QuivOR™ is adequately secured to the sterile patient drapes prior to inserting any instrumentation.
- The QuivOR™ will comfortably accommodate up to three (3) separate surgical instruments.
- After the procedure discard the device in the approved medical waste container in accordance with institutional policy.

Any serious incident that has occurred in relation to this device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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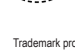
| | | | |
|---|--|---|--|
|  | Product conforms with the requirements of European Medical Device Regulation 2017/745. |  | Caution: US Federal Law restricts this device to sale by or on the order of a physician. |
|  | Caution is necessary when operating the device. |  | Do not use if package is damaged. |
|  | Do not re-sterilize. |  | Consult Instructions for Use. |
|  | Keep away from sunlight. |  | Not made with natural rubber latex. |
|  | Keep dry. |  | Do not reuse. |
|  | Single Sterile Barrier System. |  | Medical Device. |
|  | Method of sterilization using ethylene oxide. |  | Trademark protected. |

Distributed exclusively in USA by: Symmetry Surgical, 3034 Owen Drive, Antioch, TN 37013, USA
Email: customerservice@symmetryurgical.com, Telephone: USA & CANADA +1 800-251-3000 (Monday-Friday, 7:00AM - 5:30PM CST)

The O R Company Pty Ltd, 447 Wangara Road, Cheltenham, Victoria 3192, Australia.
www.theorcompany.com

EC REP OBELIS SA
Bd Général Wahis 53, 1030 Brussels, BELGIUM, Tel: +(32) 2 7325954

Issued Date: 08-03-2022

PM_QVOR_101_V4