Symmetry Surgical® BOOKWALTER® Retractor Segmented Rings

Symmetry Surgical Inc.
3034 Owen Drive
Antioch, TN 37013 USA
☎ 1-800-251-3000
FAX: 1-615-964-5566
www.symmetrysurgical.com

Symmetry Surgical GmbH
Maybachstraße 10
78532 Tuttlingen, Germany
☎ +49 7461 96490
Fax: + 49 7461 77921

LCN 500-4583-85/P

© 2002–2015 Symmetry Surgical
♻ Revised 09/15

ENGLISH

IMPORTANT INFORMATION

Please Read Before Use

Symmetry Surgical® BOOKWALTER® Retractor Segmented Rings

Rx Only

Description

The Symmetry Surgical BOOKWALTER® (Figure 1) segmented rings are composed of four interchangeable components: two half circles and two straight ring extensions. The segments can be assembled to form a round ring, an oval ring, and in the case of the elliptical segmented ring, an ellipse. The size and shape of the rings may be adjusted intraoperatively by angling a section out of the way or removing it. The components of all of the rings are interchangeable. The device is provided nonsterile and must be sterilized before use, including first use.

BOOKWALTER® Retractor Rings are only available from Symmetry Surgical. Contact your Symmetry Surgical sales representative for more information on ordering additional rings.

The rings come in four sizes: small (Catalog No. 50-4587), medium (Catalog No. 50-4583), large (Catalog No. 50-4588), and elliptical (Catalog No. 50-4595). Each of the rings comes with one set of straight segment lengths. The rings have the following

dimensions, depending upon which set of straight segments is used:

Size	Internal Diameter		
	4 inch (10.16 cm) segment	6 inch (15.24 cm) segment	8 inch (20.32 cm) segment
Small (50-4587)	14 x 8.5 inches (35.6 x 21.6 cm) (Shipped standard with the small rings)	16 x 8.5 inches (40.6 x 21.6 cm)	18 x 8.5 inches (44.5 x 21.6 cm)
Medium (50-4583)	16 x 10.5 inches (40.6 x 26.6 cm)	18 x 10.5 inches (45.7 x 26.7 cm)	20 x 10.5 inches (50.8 x 26.7 cm) (Shipped standard with the medium rings)
Large (50-4588)	18 x 12.5 inches (45.7 x 31.8 cm)	20 x 12.5 inches (50.8 x 31.8 cm) (Shipped standard with the large rings)	22 x 12.5 inches (54.6 x 31.8 cm)
Elliptical (50-4595)	13 x 14.5 inches (33 x 36.8 cm)	15 x 14.5 inches (36 x 36.8 cm) (Shipped standard with the elliptical rings)	17 x 14.5 inches (43 x 36.8 cm)

The three straight segment lengths can be used with any of the rings. These straight segments have the following catalog numbers:

- 4 inches (10.16 cm) – Catalog No. 50-4636
- 6 inches (15.24 cm) – Catalog No. 50-4637
- 8 inches (20.32 cm) – Catalog No. 50-4638

Precautions

If using monopolar electrosurgical equipment while using the BOOKWALTER® Retractor Kit, consult the basic rules of usage concerning electrosurgery (see EN 60-601-2-2), as well as the manufacturer's instructions for use regarding specific warnings and conditions.

Before fully tightening the locking screw, always be certain that the serrated teeth of the ring segment fittings mesh. Failure to do so can damage fittings.

Be sure to electrically isolate the patient from high frequency leakage currents to the operating table, in accordance with the instructions provided by the manufacturer of the electrosurgical equipment.

Ensure that the patient does not come into contact with the metal parts that are earthed or that have an appreciable capacitance to earth (for example: operating table supports), in accordance with the instructions provided by the manufacturer of the electrosurgical equipment.

Avoid small-area contacts with objects having a low impedance to earth at high frequencies, as unwanted burns can result.

Indications

The Symmetry Surgical BOOKWALTER® Retractor Segmented Rings are designed to be used with the BOOKWALTER® Retractor System. Please refer to the instruction manual provided with the retractor.

Using the Segmented Ring

Center the ring over the operative site and adjust the BOOKWALTER® Retractor post coupling to obtain the desired position. Hold the ring in position while locking in place. Make any final positioning adjustments with the ring attached over the wound.

To remove or adjust a ring segment, turn the locking screw counterclockwise until the serrated teeth of the fittings disengage. Adjust the segment as required and retighten the locking screw.

CAUTION: Before fully tightening the locking screw, always be certain that the serrated teeth of the ring segment fittings mesh. Failure to do so can damage fittings.

To remove the screw from a ring segment for cleaning, pull the screw through the fitting until it engages the threads. Unscrew and remove by turning clockwise.

Inspection

Inspect each instrument before and after use to detect imperfections. Do not use damaged instruments; send them for repair or replacement.

Reprocessing instructions

Refer to Symmetry Surgical Instructions for Use #204233 for updated information regarding reprocessing the instrument.

Copies of this instruction sheet are available by requesting IFU #204233 from Symmetry Surgical Customer Service at:

Telephone: 800 251 3000
Fax: +1 615 964 5566
E-Mail: customerservice@symmetrysurgical.com

Service and Repair

For service and repair outside the United States, contact your local Symmetry Surgical representative.

Inside the United States, send instruments for service or repair to:

Symmetry Surgical Inc.
3034 Owen Drive,
Antioch, TN 37013 USA

WARRANTY

Symmetry Surgical warrants that this medical device is free from defects in both materials and workmanship. **Any other express or implied warranties, including warranties of merchantability or fitness, are hereby disclaimed. Suitability for use of this medical device for any particular surgical procedure should be determined by the user in conformance with the manufacturer's instructions for use. There are no warranties that extend beyond the description on the face hereof.**

BOOKWALTER® is a registered trademark of Symmetry Surgical Inc.

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© 2002–2012 Symmetry Surgical Inc.