INSTRUCTIONS FOR USE: HolstOR PRO™ Surgical Instrument Holder

EN

INDICATIONS FOR USE / CLINICAL BENEFITS

HolstOR PRO is a surgical instrument holder, designed specifically to improve safety by reducing unnecessary instrument exchanges between surgeon and scrub personnel in the operation room.

CONTRAINDICATIONS

This device is not intended for use except as indicated.


CAUTIONS / WARNINGS

- Choose the HolstOR PRO size in relation to the surgical instruments used. Do not overload the instrument holder.
- Each HolstOR PRO requires its own towel clamp, even when connected to another HolstOR PRO.
- The device is intended for single use only. The design of this device may not perform as intended by the manufacturer if it is reused. The manufacturer cannot guarantee the performance, safety and reliability of a reprocessed device.

INSTRUCTIONS FOR USE

- Remove the HolstOR PRO from the sterile packaging and place it on the sterile instrument table.
- Drape patient as required.
- Secure the HolstOR PRO to the sterile patient drape using a towel, Kelly or artery clamp and the supplied 4mm key ring. See diagram B.
- Ensure the HolstOR PRO is adequately secured to the sterile patient drapes prior to inserting any instrumentation.
- The HolstOR PRO will comfortably accommodate up to three (3) separate surgical instruments.
- Additional HolstOR PRO instrument holders can be connected utilizing the accessory clips located on the side of the HolstOR PRO.
- After the procedure discard the device in the approved medical waste container in accordance with institutional policy.

Any serious incident that has occurred in relation to this device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

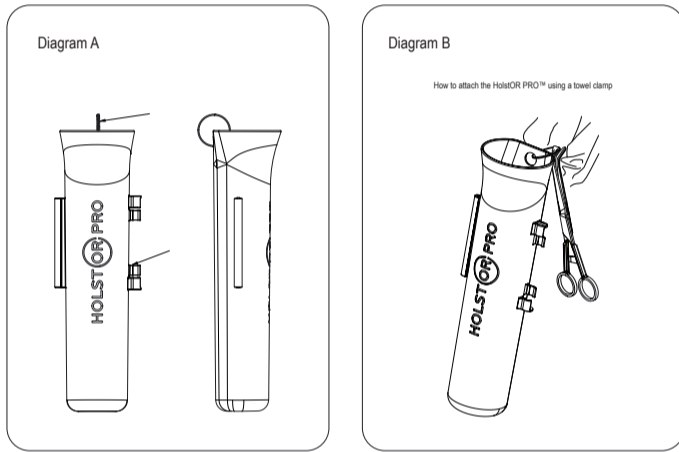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